



bobopro

International Edition Second Edition FEBUARY 2017

User Manual

THE BOBO PRO

PRODUCT INFORMATION

- <u>BOBO BOARD</u> is an affordable training board that mounts on any balance training device and turns it into a smart training tool. By tilting the board, the user controls games and training programs that motivate and challenge him.
- <u>BOBO TAB</u> A tablet-based console that runs and monitors all the BOBO games and training programs - plus hundreds of free Android 'tilt games'
- <u>BOBO CONTENT</u> offers personalized, tailor-made training programs that keep track of user performance and progress. Compatible with a wide range of free games, BOBO lets the user choose the content that s/he likes - turning exercise/rehabilitation into a fun and rewarding activity.

NOTES

- 1. Avoid the unit being dropped. Avoid the display been shake seriously, or the display will be damaged or unusual.
- 2. Do not use metal object to touch the device to prevent it from short circuit.
- Avoid the BOBO BOARD being damaged from the floor, by placing a carpet or a matress on the floor.
- 4. Do not disassemble. Do not wipe the surface of the unit by alcohol, thinner or benzene.
- Do not disconnect the unit while it is formatting or uploading/downloading, or the procedures may lead to errors.
- 6. Avoid water or foreign objects enter the unit.
- 7. Do not use the unit in places that are extremely hot, cold, dusty, or humid.
- 8. Specifications are subject to change and improvement without notice.

THE TABLET

INTRODUCTION OF COVER



OPERATION INSTRUCTION BASIC OPERATION

1. Power on/Power off

Power on

Press and hold the power switch for 3-5 seconds; the unit displays the start-up logo and enters the working mode.

Power off

Press and hold the power switch for 3-5 seconds to turn the unit off.

<u>Sleep</u>

While the device is at working status, press the power switch, it enters the sleeping mode, and the screen will be off then. Press the power switch again, it shift to the normal mode. You may use this function when you don't' use the device temporary to save power.

2. Charging

When the device prompts low battery, please charge it by provided charger.

Note: To improve the longevity of the battery, you may charge for another 15 minutes after it prompts the battery is full.

SETTINGS

Click "Settings" in the main interface to enter the setting interface. Click the corresponding setting items to operate.

1. Date and Time Setting

Click "Date & time" in the setting interface to set the date and time. You may set the date, time zone, time, time format and date format etc. in the menu.

2. Brightness

Select "Display" in the setting interface. Click "Brightness" to enter, and slide the bar to adjust the brightness.

3. Screen Timeout

Adjust the delay before the screen automatically turns off.

Select "Display" in the setting interface. Click "Sleep" to enter, and click the time you want. Note: The setting of screen timeout is unavailable while playing video.

WI-FI CONNECTION

Click "Settings" in the main interface. Click Wi-Fi to turn on the WIFI, it lists the open network it searched. Click the name of the network; it pops up the setting window. You may connect it to WIFI according to the prompts. You need to input the user name and password if networks are encrypted.

1. Bluetooth

Turn on the Bluetooth, Click Bluetooth setting to set and manage Bluetooth Devices access.

Click "SEARCH FOR DEVICES", it lists the open devices. Click the name of the devices; it pops up the setting window. choose the device, pairing it, then you can transfer pictures, movies, music etc with it.

2. Ethernet configuration

The Unit support USB-Net-Card

INSTALL, UNINSTALL AND RUN APPLICATIONS

1. Install Apps

There are many ways to install applications under Android, such as via USB, by third party program. We introduce installation by Google Play and BOBO INDEX.

Click the "Google Play" icon in the main interface. Login to your Google account - on Google Play store, Search "BOBO INDEX" and install it - Select the APK file you want from the "BOBO INDEX" and enter the application installation interface. Click "Install" to install the application automatically. After installation, it prompts" Open" and "Finish". Click "Open" to open the application. Click "Finish" to exit.

2. Uninstall

You may use the corresponding third party software management tool or the installed application management software of the device to install applications.

Select "Setting / Applications / Manage applications" to select the installed applications and check the attribute of it or uninstall it.

CONNECT TO INTERNET

1. After connecting with WIFI, open the browser and input the web address, you can browse the web then.

2. Click the 📩 icon to enter the bookmark to view the browse history. Click the web page you want to open it.

AUDIO AND VIDEO PLAYBACK

1. Audio playback

Click the "MUSIC" in the main menu to enter the audio playback interface. It lists the audio files in the device, and you may click the file name to play it. in the audio playing interface, click the icons to play/ pause / increase volume / decrease volume / FF / REW.

2. Video Playback

Click the "Gallery" in the main menu to enter the video playback interface. It lists the video files in the device, and you may click the file name to play it. in the video playing interface, touch the screen to pop-up the operation icons and click the icons to play / pause / increase volume / decrease volume / skip to next file / skip to previous file.

PICTURE VIEWING

Click "Gallery" in the main menu to enter the picture folder list. Select and click the folder you want to enter, it displays the . click the thumbnail of the picture to view it.

Touch the screen to pop-up the operation icons while viewing the picture, you may zoom in/zoom out/ skip to next picture/skip to previous picture / rotate / delete the picture...

CAMERA

Click "Camera" in the main interface.

1. Camera

If the cursor is at 😐 📑 position, the device is at Camera mode.

- Click 🧕 to take picture.
- Click 📑 to set the scene mode, Picture size, Picture quality, and colour effect.
- Click or to set white balance.

2. DV

If the cursor is at a position, the device is at DV mode.

Click 💽 to take picture.

Click 😅 to set the colour effect or restore to default settings.

CONNECT WITH EXTERNAL DEVICE

1. Connect with PC

Connect the device with the PC via the USB cable. The PC appears two new disk symbols after connected successfully. One is the TF card, and the other is the built-in flash.

2. Use of OTG

It supports the movable device with FAT32 format, such as U disk and MP3. Connect the movable device to the MID.

3. Applications

For convenient usage, we installed some third party software in the MID. For detailed operation methods, please check the corresponding instruction. Some of the software may be able to be used in some area because of network service.

TROUBLESHOOTING

1. What if the device is abnormal during operation?

Answer: 1) Reboot the device to see if it works OK.

2) Charge the device by the charger to activate the battery. Make sure it has power before you operate it.

3) Do not upgrade the OS of the device.

2. Why the standby time is so short sometimes?

Answer: The use time of the battery is all depend on the temperature and use instance. If the temperature is too high or too low, the use time of the battery might be effected. It is suggest using the device in normal temperature. The using time is also affected by increased volume and frequently operation.

THE BOBO BOARD



Power button

Long press to turn the power on or off. Short presses to toggle between modes.

Front

QUICK START GUIDE

1

Tablet orientation:

- Place the tablet with a landscape orientation (A). Make sure the ports and buttons are on the right hand side of the tablet.
- Go to SETTINGS > DISPLAY > WHEN DEVICE IS ROTATED and choose STAY IN CURRENT ORIENTATION.
- When playing portrait games, rotate the tablet to the right (B). Make sure the ports and buttons are on the bottom.





2

Make sure the battery is charged BEFORE you use the BOBO.

Power on the BOBO by pressing and holding the power button.

While powering on, the Power LED will pulse green.
Power LED Green:

Default mode (for landscape games).

Toggle between modes by short presses on the power button.

Power LED Blue:

Default mode (for portrait games) Toggle between modes by short presses on the power button.



3

Make sure the BOBO PRO's Bluetooth is enabled. Pairing is automatic! Do not try to pair the BOBO with the tablet through the tablet's Settings tool. If you do, the BOBO will not be able to communicate with the tablet.

Wait 10-20 seconds and verify that the egg on the bottom right corner of the tablet wallpaper responds to the BOBO's movements.

If the egg does not respond, wait another minute. Connect with a Google Play account, download and install the BOBO INDEX app



<u>4</u>

Place the BOBO on an unstable balance training platform (Don't try to use it on the floor. The BOBO will not respond without tilt movements). Example:



<u>5</u>

Position the BOBO directly in front of your TV / tablet, according to the game mode as shown below.

For LANDSCAPE games:

Power LED Green (default): Game mode facing the screen. The Power Button faces the screen. (A) <u>Power LED Blue</u>; Game mode perpendicular to the screen. The Power Button faces 90 degrees to the left of the screen. (B)

For PORTRAIT games:

Power LED Blue (default): Game mode facing the screen. The Power Button faces the screen. (A)





STAND ON THE BOBO

When placing the BOBO it on a low platform (such as a balance disc)



BOBO and apply weight.

Place one foot on one side of the

The board will tip toward that foot

until the edge of the board rests on

sturdy for support if necessary.

the around. Shift your weight entirely

onto that leg, holding onto something



2

Place your free foot on the opposite side of the board, an equal distance from the center.



Shift your weight back and forth until it's evenly balanced.

Just standing on the board like this can be a major challenge at first. Hold onto something sturdy for balance if necessary, but wean yourself off the support one finger at a time as your balance improves.

When placing the BOBO on high platforms (such as BOSU)





Place one foot in the middle of the BOBO, and slowly shift your weight onto it.



Slide your feet apart slowly one foot at a time, without lifting it off the BOBO.



4

Shift your weight back and forth until it's evenly balanced.

Place your free foot just next to your other foot.

Just standing on the board like this can be a major challenge at first. Hold onto something sturdy for balance if necessary, but wean yourself off the support one finger at a time as your balance improves.

NOW YOU'RE READY TO PLAY!

AFTER PLAYING

You can power off the BOBO by pressing and holding the power button. A long beep sounds and the Power LED turns off.

OPERATIONAL SIGNALS

Operational	Visual
Turn on, press button and hold	Status light on – GREEN
Turn off, press button and hold	Status light off
System is on and working on bobo's default mode (for landscape games)	Status light pulses GREEN
System is on and working on bobo's default mode (for portrait games)	Status light pulses BLUE
Battery low – 30 min before battery depletion	Status light pulses RED
Battery charging	Status light pulses ORANGE
Battery charged	Status light static - ORANGE

TROUBLESHOOTING

What to do if the BOBO does not connect to the tablet?

- Verify that Bluetooth is enabled on the tablet.
- Turn off the Bluetooth, wait 10 seconds and turn it on again. Wait 10-15 seconds.
- Wait 10-20 seconds and verify that the egg on the bottom right corner of the tablet wallpaper responds to the BOBO's movements. If the egg does not respond, wait another minute.Verify that Bluetooth is enabled on the tablet.
- Pairing is automatic! Make sure you didn't pair the BOBO using the tablet's Settings tool. If the BOBO was paired in this way go to SETTINGS > BLUETOOTH press on BOBOARD3 and choose the FORGET option.
- Verify that the BOBO is turned on (green or blue LED blinking).
- Turn off the BOBO and reset the tablet. Wait until the home screen re-appears. Now, power on the BOBO again and wait for a minute.
- Make sure that the BOBO's battery is charged by connecting the charger cable to the BOBO and checking the charging status.

What should I do if BOBO'S wallpaper has disappeared?

Long press on the desktop, choose WALLPAPERS and then choose BOBO WALLPAPER

What should I do if the tablet screen orientation changes frequently while using the BOBO?

- Turn off the BOBO.
- Hold the tablet in the correct landscape position (buttons and ports face to the right).
- After 15 seconds the screen orientation will correct itself automatically.
- Go to SETTINGS > DISPLAY > WHEN DEVICE IS ROTATED and choose STAY IN CURRENT ORIENTATION.

What should I do if the tablet screen doesn't appear on the TV screen?

- Make sure the HDMI cable is connected properly at both ends tablet and TV screen.
- Make sure that HDMI is enabled on the tablet: Go to SETTINGS > DISPLAY > HDMI SETTINGS. Verify that HDMI is ON.
- Make sure that you chose the correct source on the TV.

What should I do if the Status Light is off?

- Try to power on the BOBO by pressing and holding the power button.
- Charge the BOBO, ensuring the cable is properly connected to the USB port. The status light should pulse ORANGE while charging.
- The BOBO does not work while charging!

SUPPORT

For technical support you can write us directly at: info@bobo-balance.com

For any other issues, contact us through our website: www.bobo-balance.com

COPYRIGHT

All Rights Reserved BO&BO Ltd. 2016.

WHAT'S IN THE BOX?

Features +	BOBO	Wireless connection to BOBO TAB
Specifications		 14 hours of continuous use without recharging
		LED & audio indicators
		Sensors: 3-axis accelerometer
		 Operating Temperature: -5°C to 45°C (23°F to 113°F)
		 Battery Type: Rechargeable 100mAh lithium polymer
		Communication Modules: Bluetooth low energy (BLE)
	BOBO TAB	OS: Android
		Screen: 10.1" touchscreen
		4 hours of continuous use without recharging
		Processor: Quad core, 1.3 GHz
		Graphic Processor: Quad core
		Internal Memory: 8GB
		BAM: DDB3 1GB
		External capacity: TF card, up to 32GB
		Audio: MP3, WMA, FLAC, APP, OGG (support all audio formats)
		 Video: MPEG4-sp, asp ,divx, WMV, RMVB, H.264/1080P (support all video formats)
		Temperature: -20°C to 60°C
		 Power supply: AC 100~240V 50/60 Hz; DC 5V/2A
		Battery: poly-lithium battery
In the Box		BOBO + BOBO Wall Mount Kit
		BOBO TAB + BOBO TAB Case Wall / Table Mount
		Quick Start guide
		Cables: HDMI, DC, micro USB
		BOBO Wall mount Guide

MEANING OF THE DUSTBIN SYMBOL



Protect our environment: do not dispose of electrical equipment in your domestic waste. Please return any electrical equipment that you will no longer use to the collection points provided for their disposal. These simple steps can help to avoid the potential effects that incorrect waste disposal can cause on the environment and human health. This will contribute to recycling and other forms of re-use of electrical and electronic equipment. Information concerning where the equipment can be disposed of can be obtained from your local authorities.



MADE IN CHINA

INFORMATION AND DIRECTION FOR USE

Please observe the following precautions before using your BOBO:

 Inspect the unstable balance platform under the BOBO prior to each use for nicks, tears, punctures or other defects that may cause the unstable balance platform to fail. Do not use an unstable balance platform if damage, punctures or defects are found.

 After placing the BOBO at the center of the chosen unstable balance device, make sure that the BOBO is firmly attached to the balance device surface. Do not use the BOBO If the BOBO glides or slips from the device surface.

• Always use the BOBO and the unstable balance platform on a flat, horizontal and stable surface.

Use the BOBO in an open space away from furniture, walls and equipment. Inspect the floor surface for any small objects that could puncture or damage the unstable balance platform.

- During practice, make sure that you have a support such as a wall or a stable table within an arm's reach.
- . Do not engage in exercises that require bouncing or jumping while using the BOBO.
- . No more than one person should use the BOBO at a time.
- Avoid stepping or standing on the edges of the board.
- . Do not use tape, nails or similar devices to attach the BOBO to the unstable balance platform.
- . Do not expose the board to liquids or to extremes of heat or cold.

 Do not use the BOBO if you are under the influence of alcohol or any other substance that could affect your sense of balance or perception and cause you to slip or fall.

The life expectancy of this product depends on how much it is used. As a precaution, the BOBO should be
inspected frequently and replaced as necessary.

 An adult must explain all Safety Information and usage guidelines and instructions to all children or people with mental or cognitive problem who are going to use the BOBO. Failure to do so may result in injuries or damage to the system or other property. Observe children during gameplay for proper and safe use of the board.

. An adult should initially assist young children to make sure they understand correct usage.

B0&B0 LTD. ASSUMES NO LIABILITY FOR INJURY OR DAMAGES THAT MAY OCCUR WITH THE USE OF THIS PRODUCT. THE CONSUMER ASSUMES ALL LIABILITY AND USES THIS PRODUCT AT THEIR OWN RISK

IMPORTANT SAFETY INSTRUCTIONS

. Do not alter, disassemble, open, crush, place pressure or crush the product.

- . The product includes lithium ion battery. Keep away from fire.
- . Keep the product dry. Do not make use with wet hands. Moisture can cause electric shock.
- . The product may heat up during use This is normal.
- Do not use or leave the battery near Fire, Oven, direct exposure to sunlight, car parking in the sun or any other heat source. Doing so may cause the battery to overheat and lead to torching.

 If the product begins to swell or expand, distributor a smell or heats up to the stage you cannot touch it, stop using the product immediately and contact a service laboratory.

 Do not make any changes / corrections independently use help by only by a qualified technician laboratory importer / manufacturer.

- If the battery starts to spread liquid, stop using the product immediately and contact a service laboratory.
- Ensure that the use is in accordance with the manufacturer's instructions.
- . Do not leave the unit in charge mode for a long time.
- . Do not store the product in a hot or humid environment.

Use only the specified charger product. Using incorrect charger may result in overheating, fire and explosion
of the battery.

 Beware of exposing the battery to water or any other liquid, not for cleaning the contacts and no other purpose.

 Do not insert or place the device in to or next to a microwave, oven or any appliance that generates heat, due to the risk of overheating of the battery.

If the battery is not charging effectively, please avoid repetition of long, continuous charging excessively.
 Reload extended without discharging the battery may cause overheating.

- Battery should not spread a strange smell or fluid material if so please call service center.
- . The battery contains combustible material and may catch fire.

 Never leave the product in areas with direct exposure to sunlight, a car parked in the sun or in any environment with high temperature.