

# Stainless Steel Finger Goniometers

## User Manual

### Intended Purpose

A finger goniometer measures range-of-motion (ROM) of finger joints (metacarpophalangeal and interphalangeal) and other small joints including toes.



### Standard, 6 inch model

Standard 6" stainless steel finger goniometer has head with opposing 180° to 30° scales in 5° increments. Linear scale on arm measures in inches and centimeters.



### Deluxe, 6 inch model

Deluxe 6" stainless steel finger goniometer has extra-large head with opposing 180° to 0 scales in 5° increments. Linear scale on arm measures in inches and centimeters.



### Small, 3.5 inch model

Small 3½" stainless steel finger goniometer has head with opposing 180° to 30° scales in 5° increments.



|         |                                       |
|---------|---------------------------------------|
| 12-1010 | SS finger goniometer, 6 inches        |
| 12-1011 | SS deluxe finger goniometer, 6 inches |
| 12-1015 | SS finger goniometer, 3-1/2 inches    |

### Measurement Tolerance

This device measures inches and centimeters with a tolerance of  $\pm 0.04$  inches ( $\pm 1$  millimeter) and degrees with a tolerance of  $\pm 1$  degree.

### Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.

### ⚠ WARNING:

- Clean and disinfect device before and after each patient use
- Verify measurement markings are clearly visible
- Ensure smooth operation of all moving parts
- Align device properly with anatomical landmarks
- Apply gentle pressure during measurements
- Stop if patient reports pain or discomfort
- Replace if accuracy becomes compromised
- Do not attempt to bend or modify the device



Fabrication Enterprises Inc  
250 Clearbrook Road, Suite 240  
Elmsford, NY 10523 (USA)  
tel: +1-914-345-9300 • 800-431-2830  
fax: +1-914-345-9800 • 800-634-5370  
FabEnt.com



AJW Technology Consulting GmbH  
Breite Strasse 3  
40213 Düsseldorf Germany



MedEnvoy  
NL-IM-00000248  
Prinses Margrietplantsoen 33, Suite 123  
2595 AM, The Hague (NL)