

Large Joint, Stainless Steel Goniometer

User Manual

Intended Purpose

A goniometer measures range-of-motion (ROM) of various joints. Use to measure large joints such as the shoulders, hips and knees.

The 180° head is marked in 1° increments. Arm measures in inches and centimeters. Tension between the arms is controlled by the thumb knob.

How to Use

1. Place the goniometer on the joint (body part) being measured.
2. Guide the joint through its full range of motion.
3. Record the result shown on the scale of the goniometer.



Measurement Tolerance

This device measures inches and centimeters with a tolerance of ± 0.04 inches (± 1 millimeter) and degrees with a tolerance of ± 1 degree.

Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.

WARNING:

- Clean and disinfect device before and after each patient use
- Verify measurement markings are clearly visible
- Ensure smooth operation of all moving parts
- Align device properly with anatomical landmarks
- Apply gentle pressure during measurements
- Stop if patient reports pain or discomfort
- Replace if accuracy becomes compromised
- Do not attempt to bend or modify the device



Fabrication Enterprises Inc
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



2862



AJW Technology Consulting GmbH
Breite Strasse 3
40213 Dusseldorf Germany



MedEnvoy
NL-IM-000000248
Prinses Margrietplantsoen 33, Suite 123
2595 AM, The Hague (NL)