


Gravity Inclinator

Instructions for Use

Intended Purpose:

The Baseline[®] Gravity Inclinator is used to measure range-of-motion (ROM).

Use: Place inclinometer near joint to be measured; turn dial until scale reads 0; take joint through its range; read range traveled directly from dial. Some neck and back measurement protocols require the simultaneous use of 2 inclinometers.

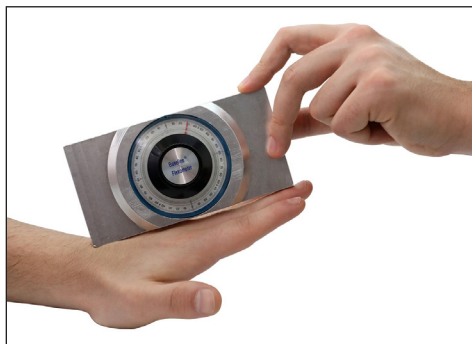

⚠ WARNING:

- Verify calibration before each use
- Clean device between patients
- Zero device before measurement
- Apply consistent gentle pressure
- Store in protective case

Measurement Tolerance

This device measures degrees with a tolerance of ± 1 degree.

Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.



Fabrication Enterprises Inc
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



AJW Technology Consulting GmbH
Breite Strasse 3
40213 Dusseldorf Germany



MedEnvoy
NL-IM-000000248
Prinses Margrietplantsoen 33, Suite 123
2595 AM, The Hague (NL)