


Universal Goniometers

Instructions for Use

Intended Purpose: Universal Goniometers are used to measure range-of-motion (ROM). Instrument available with clip to fasten goniometer to subject, or headband.


Baseline® Universal Goniometer with Clip

Use: Clip goniometer near joint to be measured. Turn dial until scale reads 0. Take joint through its range. Read range traveled directly from dial.

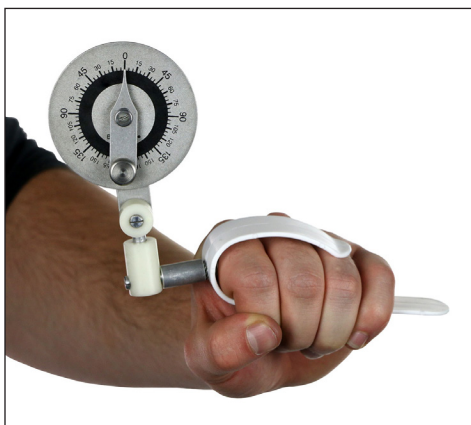

Baseline® Universal Goniometer with Headband

Use: Place headband on subject to measure ROM. Turn dial until scale reads 0. Take joint through its range. Read range traveled directly from dial.

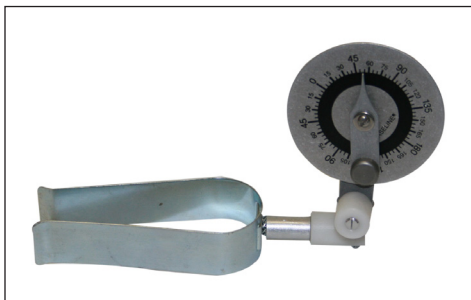

Measurement Tolerance

This device measures degrees with a tolerance of ± 1 degree.

Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.

FEI Fabrication Enterprises Inc
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



AJW Technology Consulting GmbH
Breite Strasse 3
40213 Düsseldorf Germany



MedEnvoy
NL-IM-000000248
Prinses Margrietplantsoen 33, Suite 123
2205 AM, The Hague (NL)

⚠ WARNING:

- Clean and disinfect device before and after each patient use
- Verify measurement markings are clearly visible
- Ensure smooth operation of all moving parts
- Align device properly with anatomical landmarks
- Apply gentle pressure during measurements
- Stop if patient reports pain or discomfort
- Replace if accuracy becomes compromised
- Do not attempt to bend or modify the device