


BASELINE[®] EVALUATION INSTRUMENTS

part 12-1116 skinfold caliper calibration block

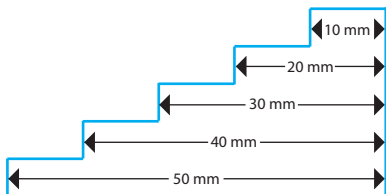

Use calibration block to check the calibration of a skinfold caliper. Open the caliper tips and gently close the tips around the 5 steps on the calibration block. Compare the caliper gauge to the length indicated on the block.



Fabrication Enterprises Inc.
250 Clearbrook Rd, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



AJW Technology Consulting GmbH
Königsallee 106
40215 Düsseldorf (Germany)