

Circumference Measuring Tape with hand-free attachment

User Manual

Intended Purpose

The Baseline[®] Measurement tape is used to measure the circumference of a joint, digit or body area. It is housed in protective plastic casing for easy opening and retracting. Woven fabric measurement tape is calibrated in inches and centimeters.

How to Use

Measurement tape has push button retractor. Measure the circumference of a joint, digit or body area. Wrap webbing around area being measured and read circumference directly from the ruler. Measure by yourself: simply make loop, position loosely around waist, wrist or area being measured, and press button. Measuring tape will stay in place securely, while you have two free hands to make adjustments and record measurements.



12-1205	circumference measurement tape, 60 inches
12-1205-20	circumference measurement tape, 60 inches (20 each)

Measurement Tolerance

This device measures inches and millimeters with a tolerance of ± 0.04 inches (± 1 millimeter)

Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.

⚠ WARNING:

- Apply consistent tension as specified in device instructions
- Clean tape surface before and after each patient use
- Do not stretch tape beyond its designated range
- Ensure tape lies flat against skin without twisting
- Replace if measurement markings become illegible
- Store flat or properly coiled to prevent damage
- Avoid sharp edges that could damage tape
- Do not use excessive force when retracting
- Verify zero alignment
- Keep retraction mechanism clean
- Replace if tape shows signs of wear