

Retractable Measuring Tape

User Manual



Intended Purpose:

These retractable measuring tapes measure objects up to 60", 72", or 120" in length in both inches and centimeters.


Introduction

The Baseline[®] Measuring tape is housed in protective plastic casing for easy opening and retracting. Woven fabric measurement tape displays inches and centimeters. Measurement tape has push button retractor.

How to Use

Pull an appropriate amount of tape out of the plastic casing. Align the zero mark (0" or 0 cm) against one side of the object you are measuring. Hold the tape down at the zero mark and pull the tape tight to the other end of the object. Record the marking you see the tape show.

Press the push-button tape retractor on the plastic casing to retract the measuring tape back into the casing when measurements are complete.

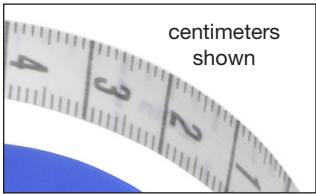

Measurement Tolerance

This device measures inches and millimeters with a tolerance of ± 0.04 inches (± 1 millimeter)

Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.

⚠ WARNING:

- Apply consistent tension as specified in device instructions
- Clean tape surface before and after each patient use
- Do not stretch tape beyond its designated range
- Ensure tape lies flat against skin without twisting
- Replace if measurement markings become illegible
- Store flat or properly coiled to prevent damage
- Avoid sharp edges that could damage tape
- Do not use excessive force when retracting
- Verify zero alignment
- Keep retraction mechanism clean
- Replace if tape shows signs of wear

12-1210	60" tape (ea)
12-1211	72" tape (ea)
12-1212	120" tape (ea)