

AffloVest® Quick Start Guide

IMPORTANT—READ THIS FIRST

THE BATTERY MUST BE FULLY CHARGED BEFORE FIRST USE

Use only the supplied rechargeable battery and the supplied power supply to ensure safe and reliable operation of AffloVest.

1 UNPACK AFFLOVEST

Unpack the AffloVest and its accessories.

2 CHECK/CHARGE THE BATTERY LEVEL

To check the battery level, depress the battery level push button to activate the battery level LEDs (Fig. 2-1). To charge the battery, connect the power supply to a wall outlet and plug the power supply plug into the battery port, into the small opening at the bottom of the battery pouch (Fig. 2-2). The power supply LED will turn on when connected to the wall outlet (Fig. 2-3).



Fig. 2-1
Battery level push button




Fig. 2-2
Power supply connected to battery







Fig. 2-3
Power supply LED lit

3 START THERAPY


Put on the AffloVest and buckle closed. To power on the AffloVest, press and hold the  button. Press **GO** button to start therapy.

NAVIGATING THE CONTROLLER

- To power On/Off. Press and hold the  (Standby) button on the controller.
- Treatments may be started at any time by pressing the  (Play/Pause) button from the main treatment menu.
 - While treatment is running, AffloVest may be paused by pressing the  (Play/Pause) button. This shall suspend treatment until the  (Play/Pause) button is pressed again.
- **GO** will automatically start the Go Mode, the system will perform the following treatment sequentially (by default):
 - Percussion Mode, 10-minute duration at medium intensity (followed by a 10 second pause).
 - Vibration Mode, 10-minute duration at medium intensity (followed by a 10 second pause).
 - Drainage Mode, 10-minute duration at medium intensity.



CONTROLLER ICON DEFINITIONS

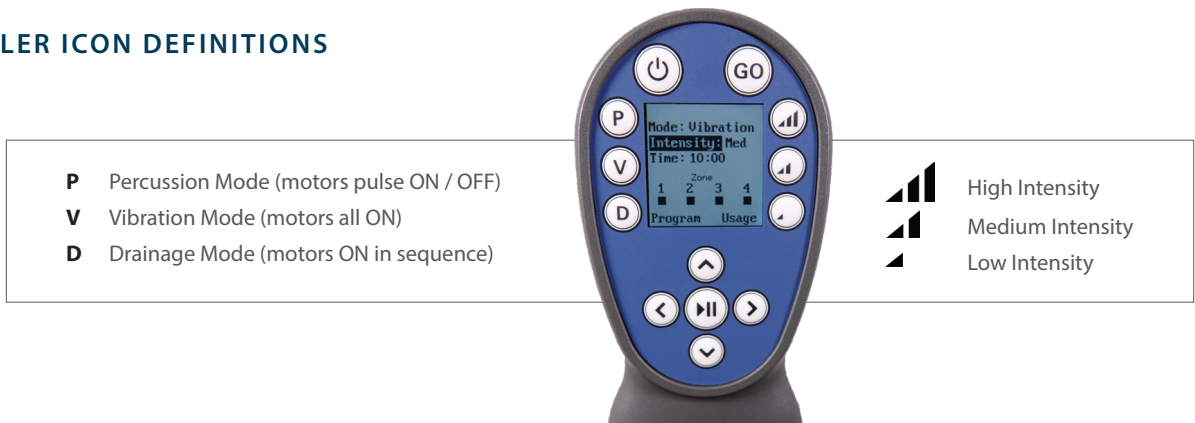

IMPORTANT TIPS FOR POWER SUPPLY AND BATTERY

- When the power supply is plugged into the battery, pay close attention not to place any strain onto the power supply cord. Rough handling of the vest and power supply while plugged into the battery could cause damage to the battery.
- One (1) LED will blink when the battery is charging. Note, this may take 5–10 seconds for the LEDs to light after plugging in the power supply.
- Although the AffloVest can be used while charging, please fully charge the battery for at least three (3) hours until all five (5) LEDs are lit solid. Once fully charged, disconnect power supply from the battery. Unplug from the wall and store the power supply until needed.
- The power supply LED will remain lit for approximately one (1) minute after unplugging the power supply from the wall.

Please read user manual for full details.

Access user manual with QR Code or visit <https://afflovest.com/manuals/>.

If you wish to order a paper copy of the user manual please call 800-575-1900 or send an email to afflovestrma@tactilemedical.com.



Feel the difference.
For more information, please visit afflovest.com.

AFFLOVEST®
by Tactile Medical

Tactile Medical

3701 Wayzata Blvd, Suite 300
Minneapolis, MN 55416 USA

Toll Free Tel: 800.575.1900
Toll Free Fax: 866.569.1912

afflovest.com