

FabStim[®] Patch

Drug Delivery System for Iontophoresis Treatments

Instructions for use

Prepare Site

To ensure best adhesion:

- Clip body hair - do not shave.
- Clean area with soap and water - dry thoroughly.

Fill Electrodes

Apply - 1.0 ml of medication to one electrode. Be sure to check polarity of medication being used for appropriate pad placement.

Start at the center of the pad and fill outward towards the edge leaving ~3 mm unfilled around the perimeter of the pad. This will keep liquid from coming into contact with the adhesive layer and compromising adhesion to the skin. Fill the opposite electrode with ~1 ml of the saline that is provided. Discard unused saline.

Place Patch

The FabStim[®] Patch should be applied to allow for full range of motion. Position joints (shoulders, elbows, wrists, knees) midway through the full range of motion to permit full movement.

Apply patch to desired location with the active pad containing the medication placed directly over desired treatment area. Pad placement will be dependent upon whether medication used is positively or negatively charged.

Secure patch by running finger around the perimeter of the outer adhesive ring. Once the patch has been fully applied, you can then press down to secure the entire patch. The patch may be taped over or wrapped for extra adhesion in high-motion areas. Ensure reinforcement is not too tight, as it can affect circulation and restrict movement.

Remove Patch

Do not pull patch off while dry. Moisten patch with soap and water to help remove the adhesive. Potential side effects to iontophoresis include skin irritation and redness around application site. You can use moisturizing lotion on the treated area following patch removal. Allow a minimum of 24 hours between FabStim[®] Patch applications on the treated area. If you are experiencing any discomfort or pain with the iontophoresis application, remove patch and notify your healthcare professional.

Rapid Release

Dosage: 80 mA/min

Fill Volume: 1.0 ml

Wear Time: ~ 4 hours

Precautions and Contraindications

- Cardiac pacemaker
- Heart conditions
- Implanted defibrillator
- Metallic or electronic implants
- History of epilepsy or seizures
- Pregnancy
- Infection
- Allergies to prescribed medication
- Open wounds or broken skin
- Compromised sensation

Indications

- Pain management
- Decrease inflammation
- Promote tissue healing



Distributed by:

Fabrication Enterprises Inc
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



Manufactured By:

Cathay Manufacturing Corp
328 Xishe Road
Maogang Town, Songjiang Area
SH 201600 ROC



LOT 1114-3 (01) 20815611021009
2022-11 (17) 221100
(10) 1111

REF

13-5272

QTY

1 each

LOT



Made in China



Rx only Federal Law restricts this device to sale by or on the order of a physician or licensed practitioner.