

INSTRUCTIONS FOR USE

Ⓞ Installing the battery

Turn the battery cover counterclockwise (↺) and remove.

Insert the C-size battery with (+) positive side facing down.

Replace the battery cover by pressing down and turning clockwise (↻).

Ⓞ Removing cap for use

Remove gray cap by pulling straight off to expose massager tip. Can also be used with cap on for wider coverage.

Ⓞ Tip insertion and removal

Select desired massage tip. Line up notch on tip base with groove on massager opening. Insert and push down as far as it will go. To remove tip, pull straight out.

Ⓞ Massaging

Remove the cap and turn the switch to the "ON" position. Tip heats and massager is ready to use on specific areas.

Before using the massager, check to see if the temperature of the heating tip is too hot to the touch. If so, place a thin

towel or handkerchief over the area to be massaged.

Ⓞ Safety tips

Do NOT use on the throat or around eyes.

Pregnant or individuals with internal disorders should NOT use massager on areas of concern.

Never use massager on infants, children or unconscious person.

Do NOT use the massager with compress or rubbing medications for skin irritations may result.

Do NOT use in room where humidity is very high such as a bathroom. This may result in wire corrosion.

Unit is NOT waterproof. Do not immerse in water or use solvents to clean. Use only a damp cloth.

Do NOT drop or leave in sun. Damage to plastic housing may result.

Do NOT leave dead battery in unit for extended period of time. It may leak and cause contact corrosion.

POINT RELIEF™ Massager with HEAT



Instructions for Use




Fabrication Enterprises, Inc.
PO Box 1500, White Plains, NY 10602 (USA)

tel: 800-431-2830 914-345-9300

fax: 800-634-5370 914-345-9800

info@FabricationEnterprises.com

PARTS GUIDE

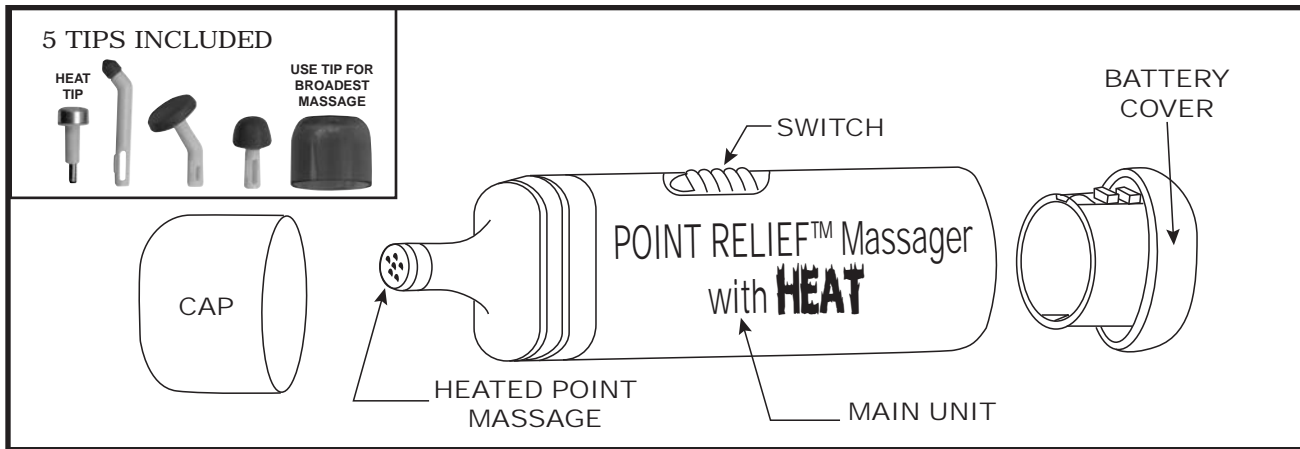

FEATURES

- Comfortable vibrating massage with heat
- Compact, portable, small enough to fit in purse or pocket
- Battery operated (uses 1 "C" battery, included)

UNIT SPECIFICATIONS

Power Source	DC 1.5V (1 "C" cell, included)
Battery Life	40 minutes of continuous operation
C.P.M.	Approximately 5,500/min
Size	1 2 ¹ / ₃₂ " x 1 1 ¹ / ₄ " x 5 3 ¹ / ₈ "
Weight	3.5 oz. without battery