
humantouch.com
800.355.2762

©2023 Human Touch®, LLC. Patents pending and intellectual property rights enforced.
No medical claims are warranted or implied by the use of this product.
Human Touch is a trademark of Human Touch®, LLC.
WBROVEPRO-NA-A0

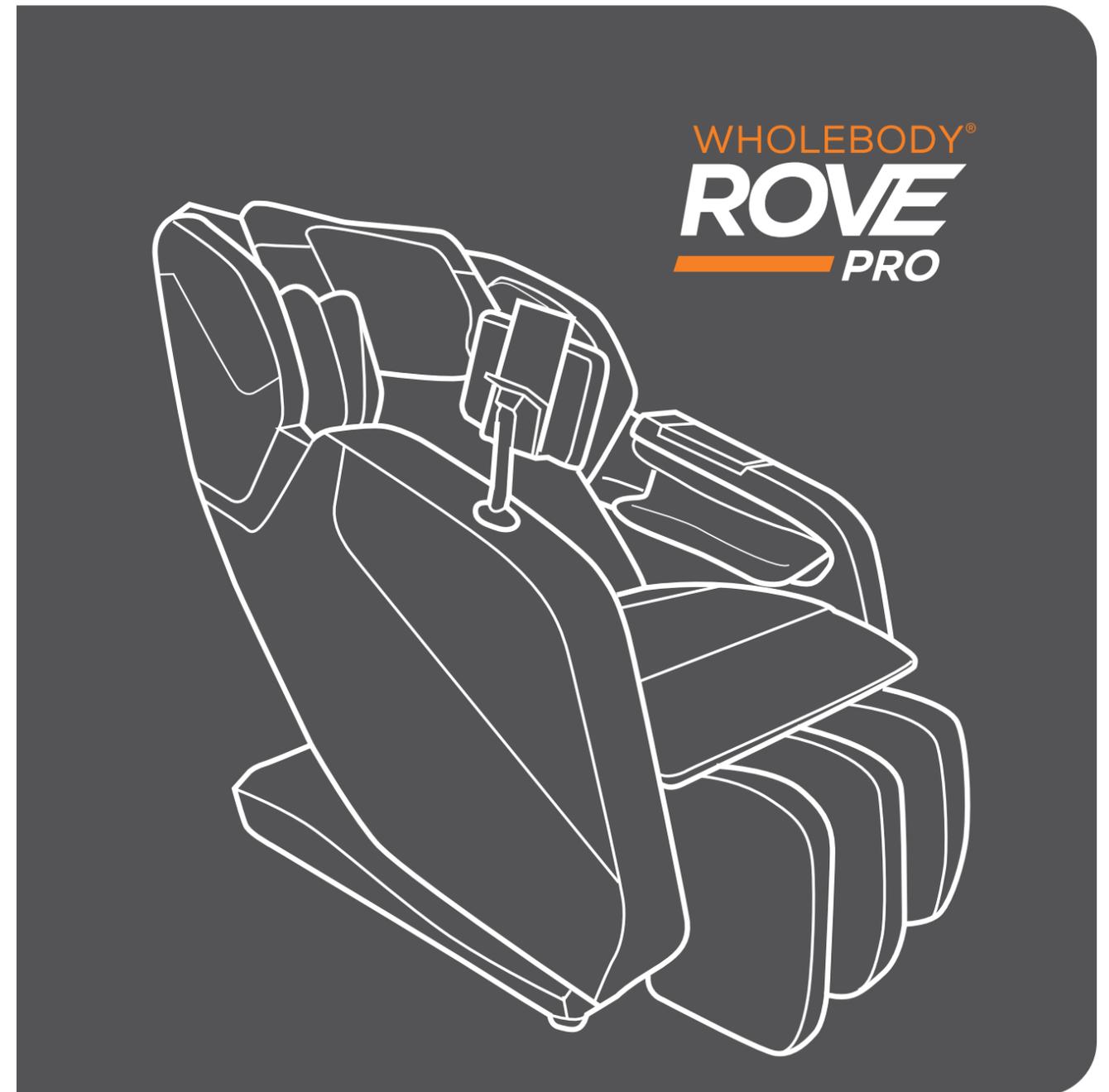use & care manual

ENG

use & care manual

FRC

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Register your product **ONLINE** today

www.humantouch.com/register

Register your product **ONLINE** today

www.humantouch.com/register

For **more than four decades**, Human Touch has established itself as an industry leader that is changing the lives of its customers with products and services that help them **feel, perform, and live their best**. Developed in cooperation with the medical community and industry experts, we deliver products that blend patented technology, craftsmanship and intelligent ergonomics with superior design to enable therapeutic relief for everyone.

Our industry-leading massage chairs, Perfect Chair® recliners and evolving set of innovative solutions share attributes in their ease-of-use, attention to performance and quality, and design to adapt to most any lifestyle. These attributes are highly valued by our customers and garner broad industry award and recognition. As a result, Human Touch empowers people to enjoy greater health and vitality in their professions, pursuits and everyday lives. And with a population increasingly aware of the benefits of wellness, Human Touch extends its brand leadership and available **solutions for the benefit of everyone**.

humantouch.com

Proudly endorsed by:

Register your product at www.humantouch.com to ensure your product is in our system to simplify the warranty process should you ever need it.

welcome to **human touch**®

Enjoy the life-changing benefits of a massage every day, keeping your mind and body healthy.

Pave your own path, make your own road, and then recover like you should. The WholeBody® ROVE PRO massage chair will make your wellness journey smooth and remarkable.

The high-performance L-track extends coverage from your neck and shoulders down to your glutes and thighs. The ROVE PRO's unique auto programs are focused on helping you relax and recover after a hard workout, or just life in general. Allow the dynamic stretch to position your body for lengthening of your back and muscles, giving you renewed energy and life—every day.

To ensure that you don't overlook any of your chair's features, and that you use it safely, read this manual carefully. Then sit back, relax, and enjoy!

humantouch.com

keep it safe

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING THIS APPLIANCE.

DANGER: To reduce the risk of electrical shock

Always unplug this product from the electrical outlet immediately after using and before cleaning.

WARNING: To reduce the risk of burns, fire, electric shock or injury to persons:

- Do not put hands or fingers near the massage mechanism or rollers while this product is running. Touching the mechanism can result in injury from the rollers squeezing together.
- If the supply cord is damaged, it must be replaced by the manufacturer or its service agent or a similarly qualified person in order to avoid a hazard.
- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
- Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- Close supervision is necessary when this product is used by, on, or near children, invalids, or disabled persons. Keep children away from calf support (or other similar parts).
- Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return this product to a service center for examination and repair.
- Always keep the power cord away from heated surfaces.
- Never operate this product with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- Never drop or insert any object into any opening.
- Do not use outdoors.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- To disconnect, turn all controls to the OFF position, then remove plug from outlet.
- Do not massage any area of the body that is swollen, inflamed or covered with skin eruptions.
- Do not use on unexplained calf pain.
- Do not use this product when standing on a damp floor, or when any part of the body is in contact with plumbing or any similar ground.
- Connect this appliance to a properly grounded outlet only. See **GROUNDING INSTRUCTIONS**.
- Do not use this product if noise above normal sound is heard.
- Do not stand on or in appliance. Use only while seated.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

SAFETY PRECAUTIONS

- Comfortable clothing is recommended while using this product.
- This product was meant to be used in a relaxed position. Never force any body part into the backrest area while the rollers are moving.
- If you feel any abnormal discomfort during the operation of this product, turn off the power immediately and do not use the product. Consult your physician.
- Do not sit on the controller.
- Do not pull on the AC cord.
- When using this product after it has been in storage, check to see that the product is operating properly prior to use.
- Always confirm that there are no obstacles behind the product and that there is sufficient space to recline.

SAVE THESE INSTRUCTIONS

Do not attempt to repair this product yourself.

This product is designed for personal in-home use. Commercial use will void warranty.


Grounding Instructions:

This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electrical current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

DANGER: Improper connection of the equipment-grounding conductor can result in a risk of electronic shock. Check with a qualified electrician or serviceman if you are in doubt as to whether this product is properly grounded. Do not modify the plug provided with the product – if it will not fit the outlet, have a proper outlet installed by a qualified electrician.

This product is for use on a normal 120-volt circuit and has a grounding plug that looks like the plug illustrated in sketch (A). A temporary adapter that looks like the adapter used in sketches (B) and (C) may be used to connect this plug to a 2-pole receptacle as shown in sketch (B), if a properly grounded plug is not available. The temporary adapter should be used only until a properly grounded outlet (sketch A) can be installed by a qualified electrician. The green colored, rigid ear tab or the like extending from the adapter must be connected to a permanent ground such as a properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by a metal screw.

For all grounded cord-connected products: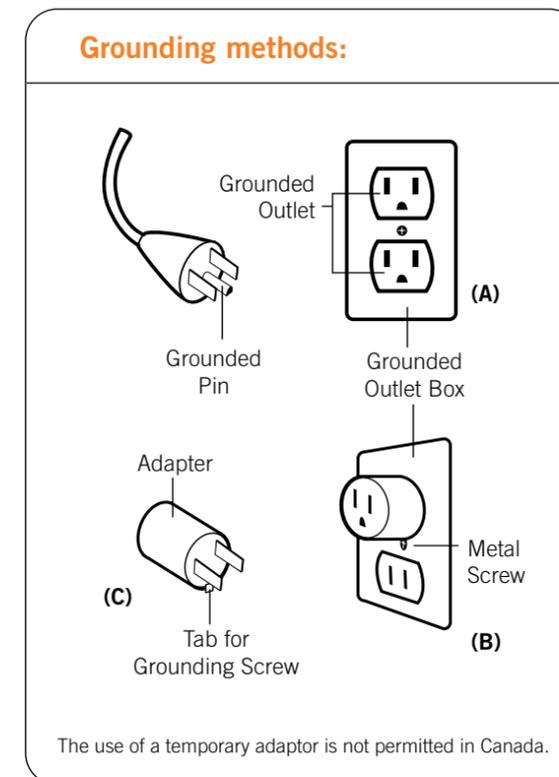


This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

WARNING: This product is equipped with a thermal resetting protection device. This is an added safety feature to protect you and this product from overheating. If the product should suddenly stop and will not start, turn the main power switch, located on the back of the product, off and do not operate the product for at least 30 minutes. Failure to turn the product off may result in the product starting unexpectedly when the device cools.


get ready

Setting up your Human Touch massage chair is easy. Just attach the calf massager, connect the body pad, and install the tablet remote and mount. Then plug it in, power it on, and you're all set!


UNPACK THE BOX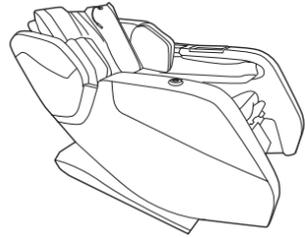

Chair Base

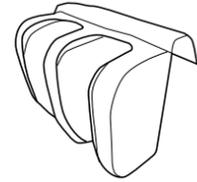Calf Massager

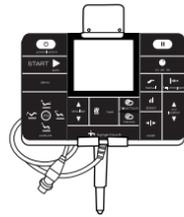Tablet Remote and Mount

Clip (extra)

Power Cord


250 V, 3.15 A Fuses (2)

CHOOSE A LOCATION

When deciding where to place your WholeBody® ROVE PRO, consider the following:



- Place the unit on a flat surface, away from direct sunlight or any other direct heat source (to avoid fading).
- If you wish to place the unit on a hard surface, it is recommended that you place a piece of carpet or a soft mat underneath it to avoid damaging the floor.
- To accommodate the calfrest when deployed, make sure the area in front of the chair is clear for at least five inches.
- The chair moves forward as it reclines, so you can place it right next to the wall.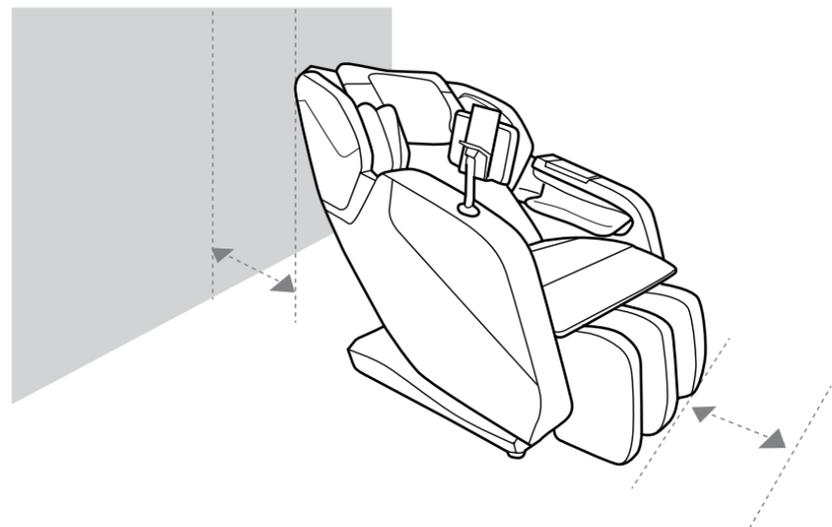

MOVE THE CHAIR TO THE DESIRED LOCATION

1. Grasp the front of the chair side panels on both sides.
2. Lift up to rotate the chair onto its wheels.
3. Roll the chair on its wheels to the desired location.
4. Gently lower the chair all the way to the floor.

WARNING: Dropping the front of the chair to the floor may damage the chair and/or floor.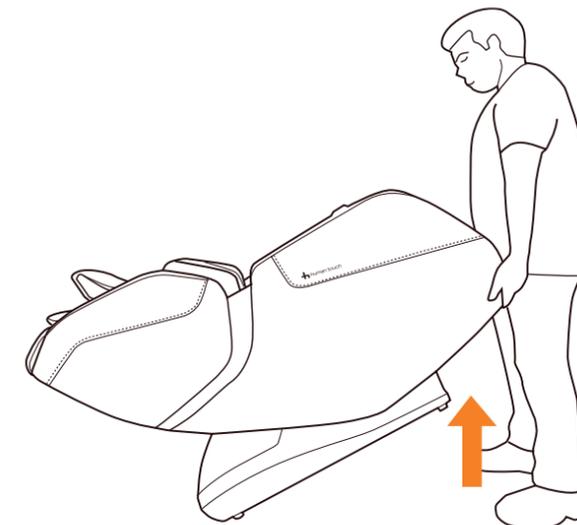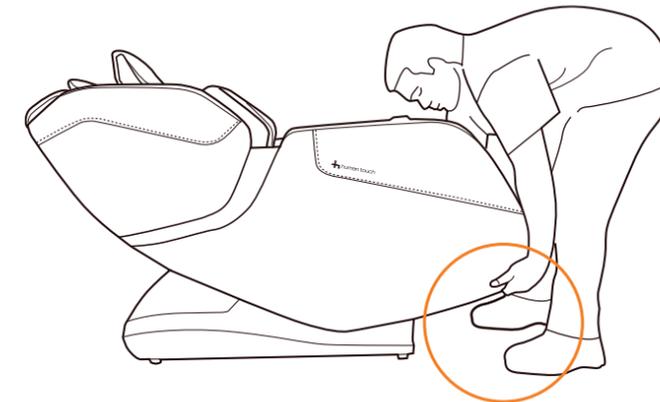

ASSEMBLE THE CHAIR

Raise the Chair to the Upright Position

1. Plug the power cord into the rear of the chair and then into a grounded wall outlet.
2. Turn the power switch located on the back of the chair base to On. The chair automatically inclines to the upright position (Fig.1).
3. Power off the chair, then unplug it.

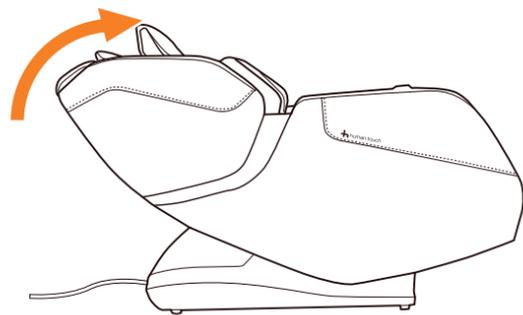
Fig.1

Install the Calf Massager

1. Remove and retain the two clips from the calf massager mounting posts located on the front of the chair by prying the tongue of each clip outward to release it (Fig.2).

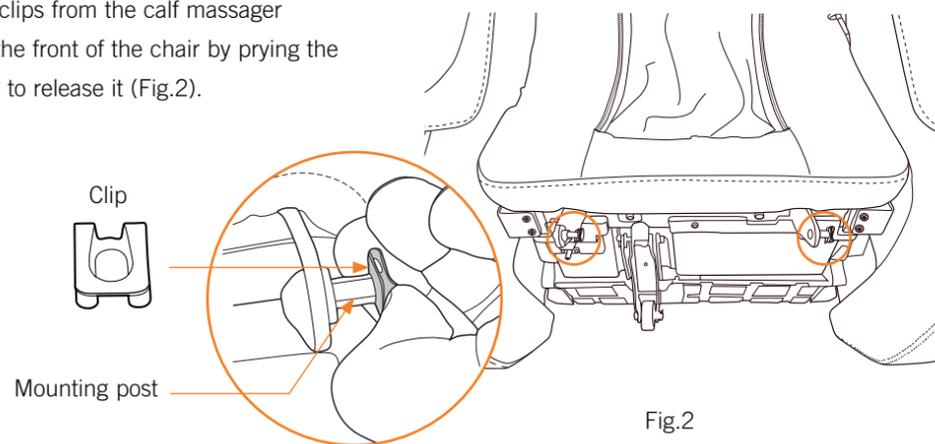
Fig.2

2. Place the massager into position in front of the chair, then connect the two air hoses and electrical connector (Fig.3).

IMPORTANT: Match the air hoses by color.

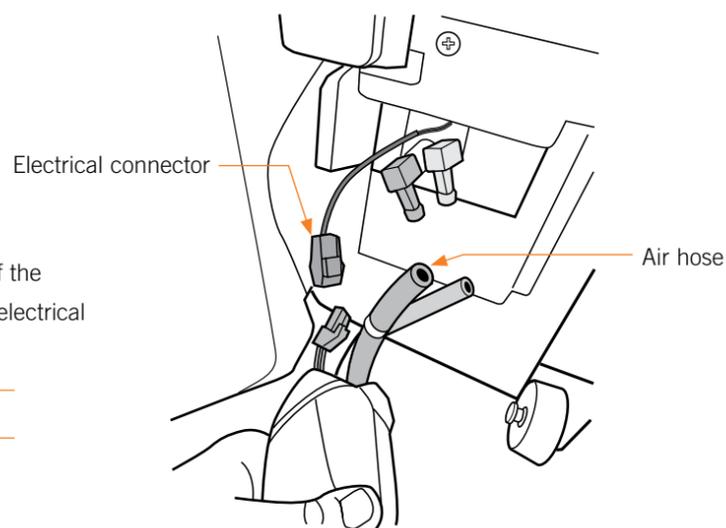
Fig.3

3. Position the massager mounting brackets to the right of the mounting posts, lift the massager from the floor, then slide the mounting brackets onto the mounting posts (Fig.4).

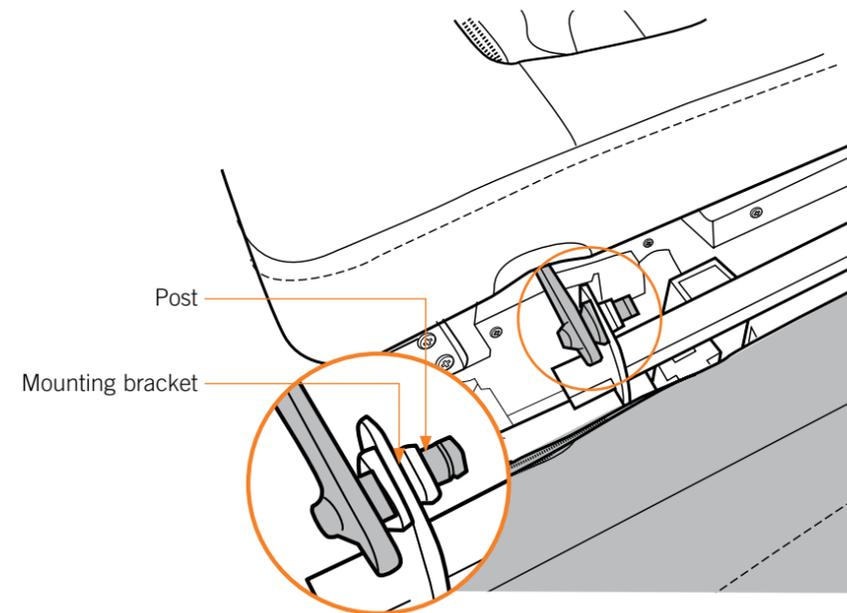
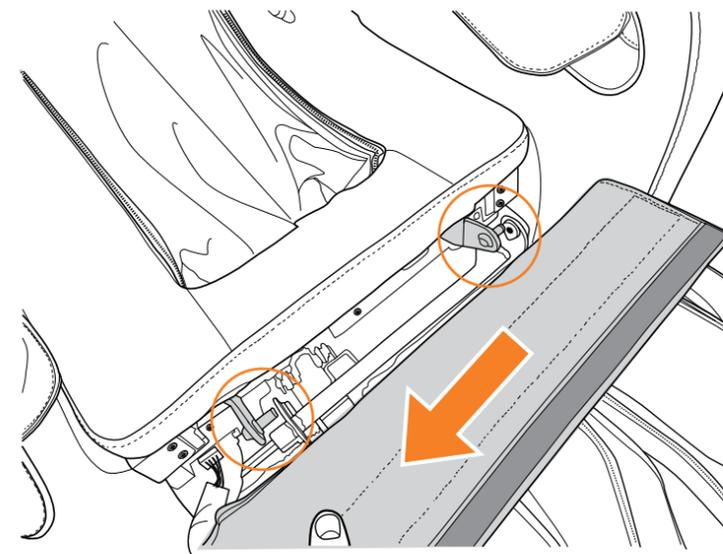
Fig.4

4. Reinstall a clip onto each mounting post with the tongue facing to the right, to secure the massager to the chair (Fig.5).

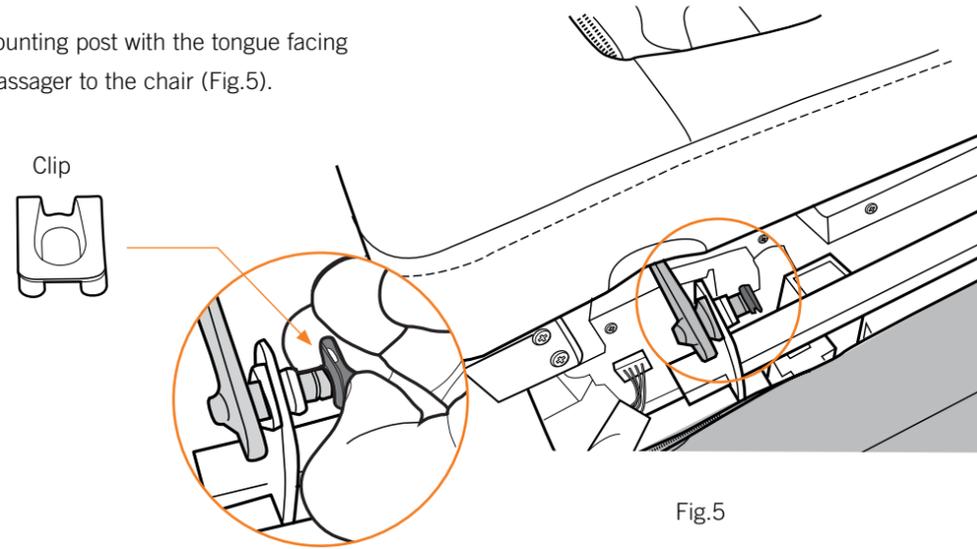
Fig.5

5. Remove the protective backing from the hook-and-loop fasteners on the back of the calf massager upholstered flap, then press the flap onto the chair base to secure it (Fig.6).

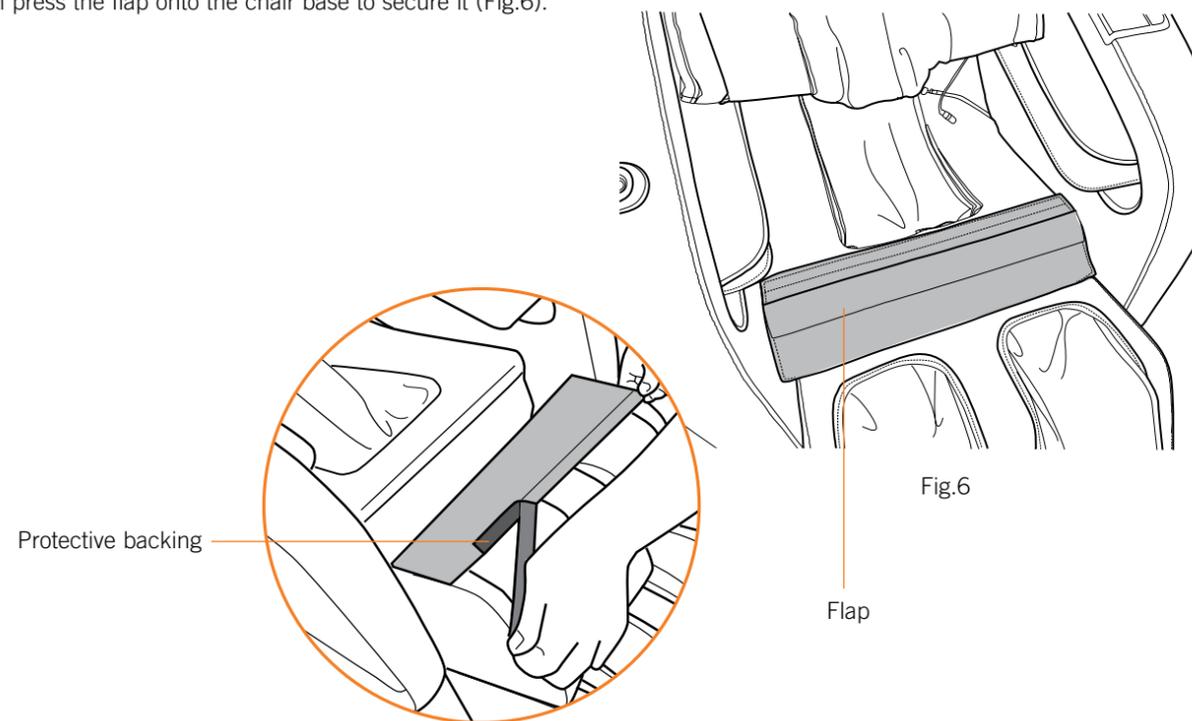
Fig.6

Install the Body Pad

1. Locate the heat cable on the left-hand side of the chair seat and the corresponding cable coming from the body pad (Fig.7).

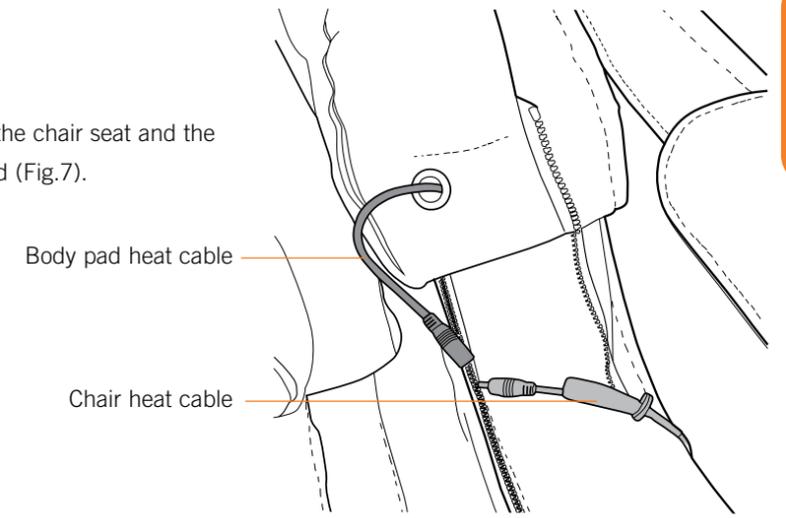
Fig.7

2. Connect the heat cables until you hear a click, indicating that the connection is properly secured (Fig.8).

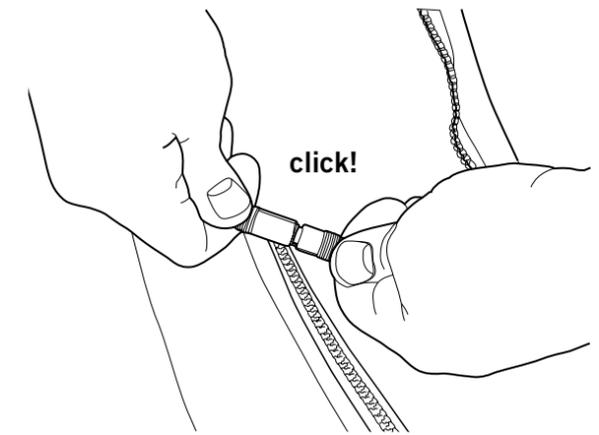
Fig.8

3. Slide the protective rubber casing over the connection to secure it (Fig.9).

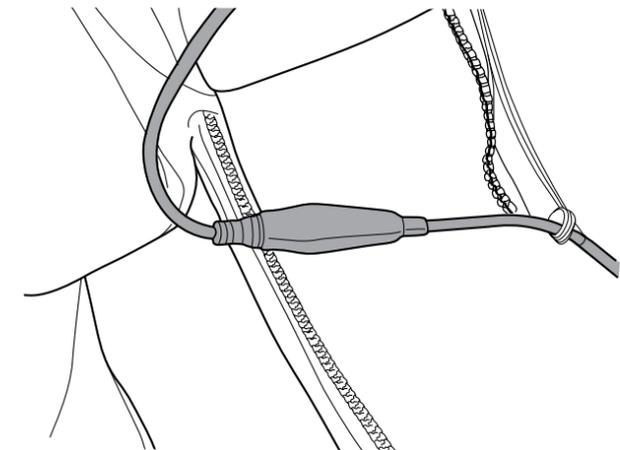
Fig.9

- 4. Zip both sides of the body pad to the chair; the zippers start at the top of the backrest (Fig.10).

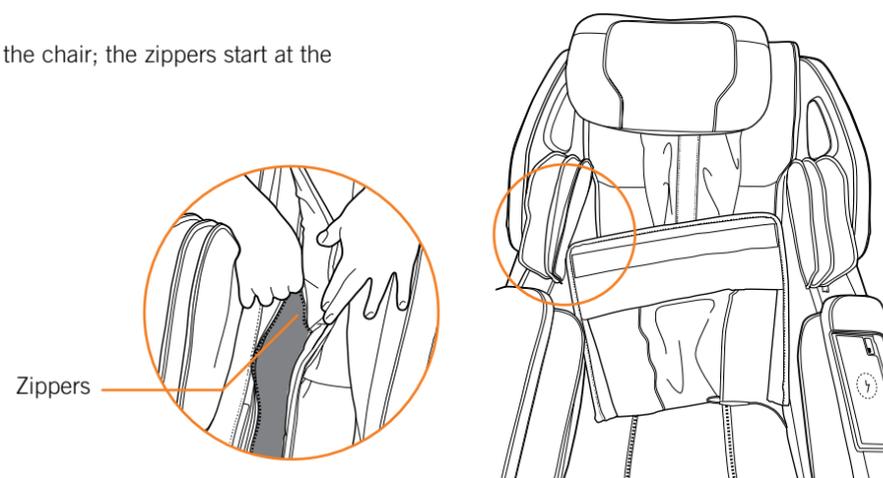
Fig.10

- 5. Remove the protective backing from the hook-and-loop fasteners on the back of the lower body pad, align the fasteners with the fasteners on the calf massager upholstered flap, then press down on the body pad to secure it to the chair (Fig.11).

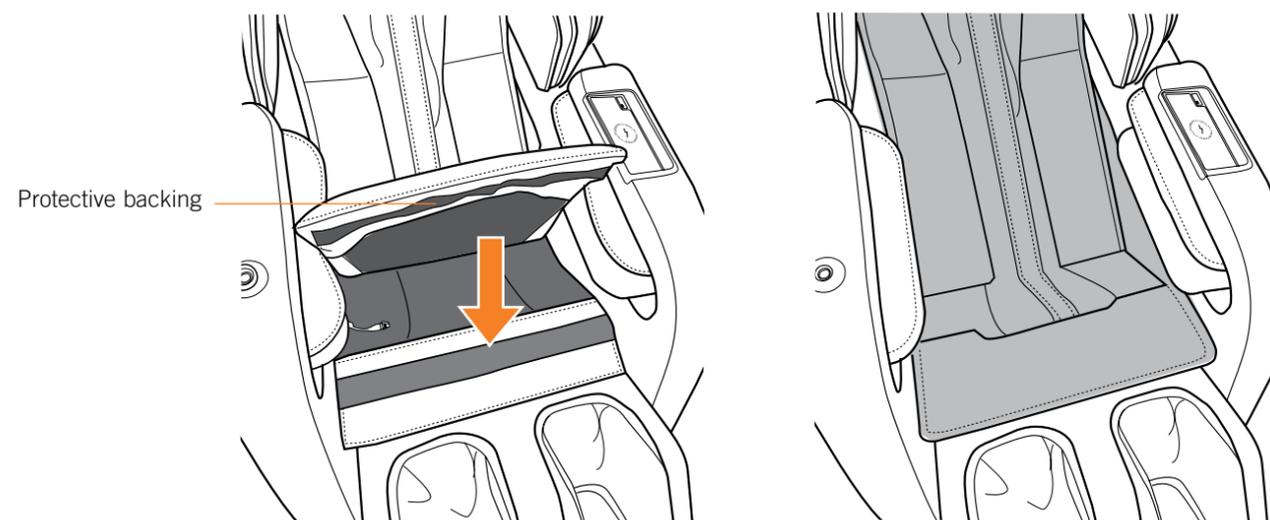
Fig.11

Connect the Tablet Remote

- 1. Locate the tablet remote connector on the right-hand side of the chair seat (Fig.12).

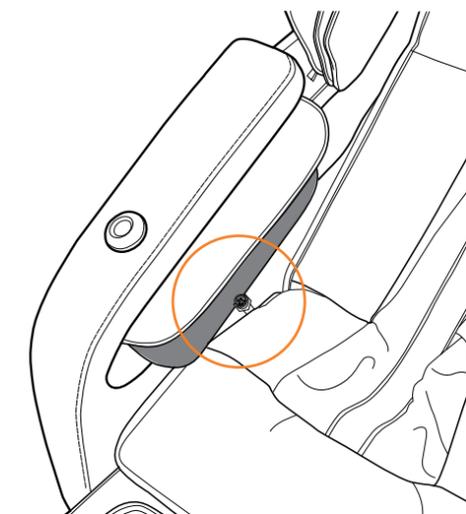
Fig.12

- 2. Matching the tablet remote connector pins with the corresponding holes in the tablet remote connector on the chair, connect the connectors (Fig.13).
- 3. Turn the connector lock clockwise to secure it (Fig.13).

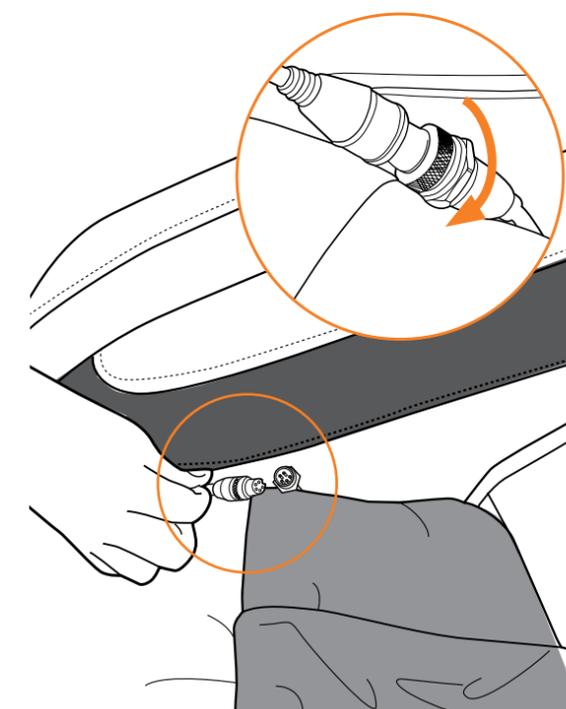
Fig.13

Install the Tablet Remote and Mount

1. Place the tablet remote mount into position with the alignment ridge facing the front of the chair, then lower the holder into the socket (Fig.14).

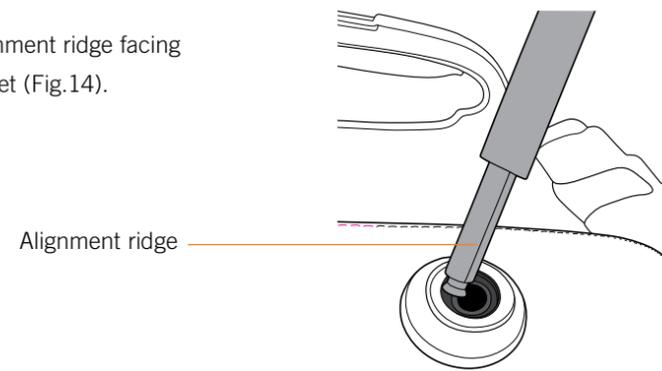
Fig.14

3. Angle the tablet remote to face you comfortably when sitting in the chair (Fig.16)

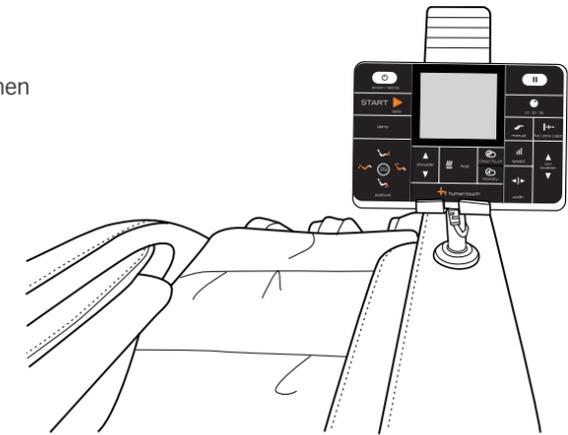
Fig.16

2. Push the tablet remote mount firmly into the socket until you hear a click (Fig.15).

Fig.15

Power the Chair On

1. Turn the power switch on the back of the chair base to On.

get acquainted

USING THE TABLET REMOTE

POWER / RESTORE — Press to activate the tablet remote. Press again to stop all function and restore the chair to the upright position.

Lights when raising or lowering the calf massager.

Press and hold to raise the calf massager.

Press and hold to recline the chair.

Press and hold to lower the calf massager.

ZEROG® — Press to recline the chair to the zero gravity position for maximum spinal relief.

Indicates the program time remaining.

Lights when in ZG position.

Lights when Pause button is pressed and all function has stopped. Screen also flashes when paused.

PAUSE — Press to pause all functions immediately. Press again to resume.

Lights when inclining or reclining the chair.

Indicates the current program in use.

Lights when manually adjusting the shoulder height.

Lights when using an auto massage program.

Flashes to indicate the massage technique(s) currently in use.

USING THE AUTO PROGRAMS

ACUPOINT® DETECTION — After selecting an auto massage program, the massage mechanism scans your back and maps the massage programs for maximum effectiveness.

AUTO — Press repeatedly to cycle through the massage programs.

DEMO — Press to start the Demo program.

MANUAL HEIGHT ADJUST — After AcuPoint Detection is complete, press and hold the arrows to adjust the vertical massage range if desired.

SELECT FROM THE FOLLOWING AUTO MASSAGE PROGRAMS.

PROGRAM	FOCUS	INTENSITY
1P	FULL BODY - Enjoy a balanced robotic and air massage from head to toe.	Medium / Low
2P	RECOVERY - Intense robotic and air massage for full-body healing to assist in recovery with greater efficiency.	Medium / High
3P	YOGA STRETCH - Relaxing full-body massage with bend and stretch functionality to provide leg and spine extension using soothing air massage and robotic massage focusing on your lumbar and glutes.	Medium / Low
4P	UPPER BODY - Relieve neck and shoulder strain with this kneading massage.	Medium
5P	LOWER BODY - Soothe your tired lower back, hips, and glutes with this kneading and tapping massage combined with air massage.	Medium
6P	RELAX - Wind down with this full-body massage.	Medium
7P	RELIEF - Relieve tension throughout the body with this primarily kneading massage.	Medium / Low
PP	DEMO - Brief, full-body demonstration of all massage features.	Medium

take control

CUSTOMIZE YOUR MESSAGE

Reads 00 when in manual mode.

CLOUD TOUCH ACCUPRESSURE — Press repeatedly to cycle through three Cloud Touch zones (Full, Upper, or Lower)

MANUAL — Press repeatedly to cycle through five massage techniques.

RANGE — In manual mode, press repeatedly to cycle through Full, Zone, or Spot massage ranges.

SPOT — When Spot massage range is selected, press and hold to adjust the vertical massage roller position.

SPEED — In manual mode, press repeatedly to adjust the massage speed (except in Rolling mode).

WIDTH — In manual mode, press repeatedly to adjust the massage roller Width (except in Kneading modes).

Indicates manual message vertical range.

Indicates message speed.

Indicates manual message width.

Indicates manual message roller width.

Indicates Cloud Touch intensity.

Indicates manual message roller width.

Fine-Tune Your Massage

After starting your massage, you can fine-tune it by setting the Duration and Cloud Touch intensity, as well as activating the heat.

Lights when backrest heat is on.

DURATION — Press repeatedly to cycle through three message timer settings (default is 20 minutes).

HEAT — Press to turn the backrest heat on/off.

CLOUD TOUCH INTENSITY — Press repeatedly to adjust the Cloud Touch intensity.

Indicates Cloud Touch intensity.

MODE	DESCRIPTION
KNEAD & PERCUSS	Combines circular kneading motion with rapid-tapping percussion strokes.
PERCUSSION	Alternates rapid-tapping strokes along the spine to stimulate both spinal muscles and spinal joint areas, emulating the percussive massage technique used in sports medicine for deep muscle therapy.
KNEADING	Kneads your back in a small, circular motion on both sides of your spine simultaneously.
ROLLING	Rolls gently up and down your back near your spine.
TAP	A pulsing and rolling massage.

want a stronger massage?

Remove the Head Pillow

For a stronger neck massage, slide the head pillow over the top of the backrest (Fig.17), or unzip and remove it.

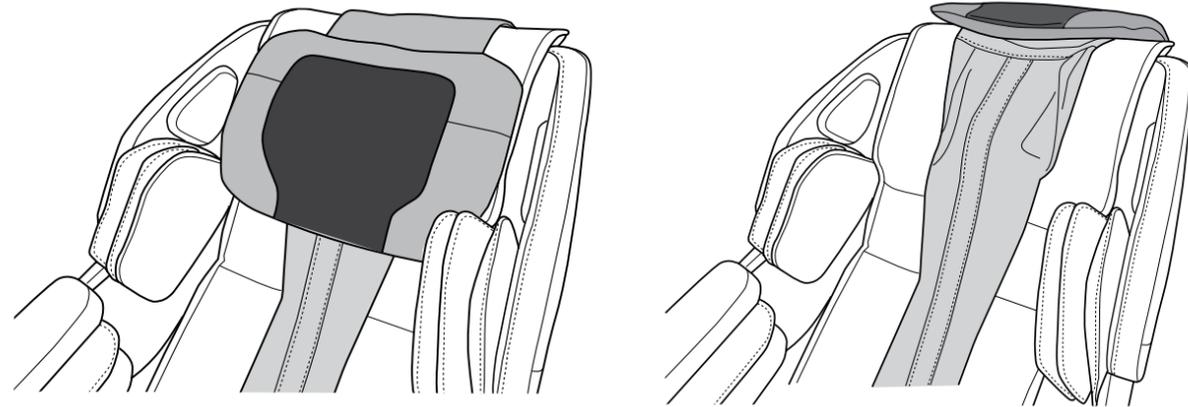
Fig.17

To remove the head pillow, open the zipper on the underside of the head pillow upholstered flap, then slide the pillow out from under the head pillow retaining strap (Fig.18).

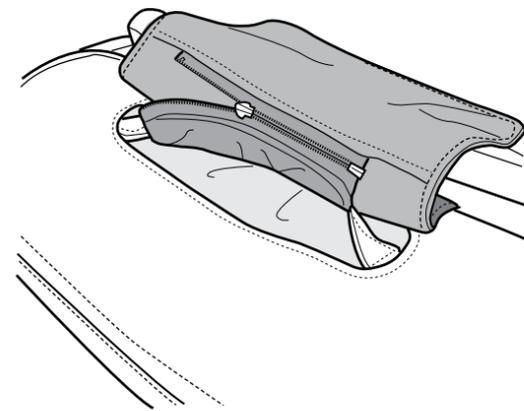
Fig.18

Remove the Softening Pad

1. To remove the softening pad, remove the head pillow, and locate the zipper on the top of the softening pad, near the back of the chair (Fig.19).

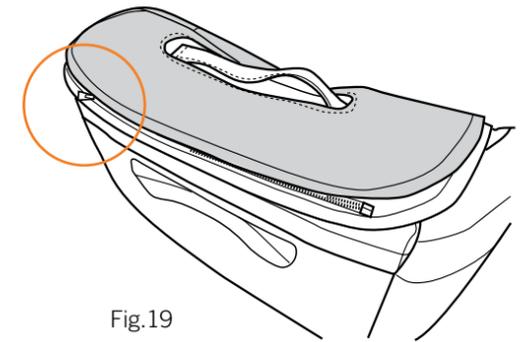
Fig.19

2. Unzip the softening pad and remove it from the chair (Fig.20).

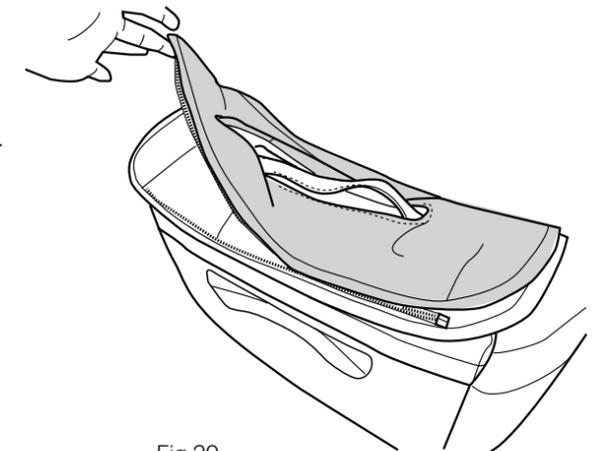
Fig.20

care and maintenance

To keep your product working and looking its best, please follow these simple care guidelines.

WARNING:

1. Be sure to disconnect the power cord from the electrical outlet before cleaning.
2. When cleaning the remote control or power switch, use a dry cloth only. Never use a damp cloth to clean any electrical component.

CAUTION:

1. Be sure to test any cleaning solution on a small, inconspicuous part of the product to ensure that the color does not fade prior to using the solution on the entire product surface.
2. Do not use benzene, thinner or any other solvents on your product.

Cleaning Code Standard for SofHyde or Cloth: W

To maintain your Human Touch product in optimum condition, dust it frequently and/or vacuum it with a soft brush vacuum extension. If your product becomes dirty, use a water-based upholstery cleaner only. To treat stains, spot clean with an upholstery shampoo or a mild detergent foam. To clean the calf massage well inserts, remove the inserts from the chair, hand-wash with a mild detergent, then line dry.

Maintenance and Repair:

1. The mechanics of this product are specially designed and constructed to be maintenance free. No periodic lubrication or servicing is required.
2. Your chair should only be serviced by a Human Touch-authorized service representative.

Use and Storage Environment:

1. Do not place the chair in direct or reflected sunlight, as ultraviolet rays will quickly fade the upholstery color and cause the SofHyde upholstery to rapidly deteriorate.
2. Operate your chair in a room temperature between 41 and 104 degrees Fahrenheit (5 to 40 degrees Celsius) with humidity levels of 10% to 70% (non-condensing) to avoid damage.
3. To prevent damage to the upholstery, before sitting in your chair make sure there are no sharp objects in your pockets and no metal buttons or zippers that can come in contact with the chair.
4. Do not place the chair where excessive dust and pet hair can migrate into the chair's electrical and mechanical components; this can cause malfunction over time.
5. When not in use for a long period of time, we recommend that the chair be covered, the power cord be coiled, and that the unit be placed in an environment free of dust and moisture.
6. Store your chair in a room temperature between 5 and 122 degrees Fahrenheit (-15 to 50 degrees Celsius) with humidity levels of 10% to 70% (non-condensing) to avoid damage; product stored in cold temperatures may require 48 hours at room temperature to function properly.
7. Do not store near open heat or flame.

In Case of a Power Surge

Your massage chair contains a built-in fuse to protect itself from a power surge. In the event of a power surge, inspect the fuse and replace it if it is blown.

To replace a fuse:

1. Power OFF the chair and unplug the chair from the wall outlet.
2. Pry the fuse holder from the compartment adjacent to the power plug input (Fig.20).
3. Replace the fuse with a new fuse of the same value as the fuse provided with the chair and listed on the fuse compartment cover, then reinsert the fuse holder into the compartment.
4. Plug the chair back into the wall outlet and power it On.

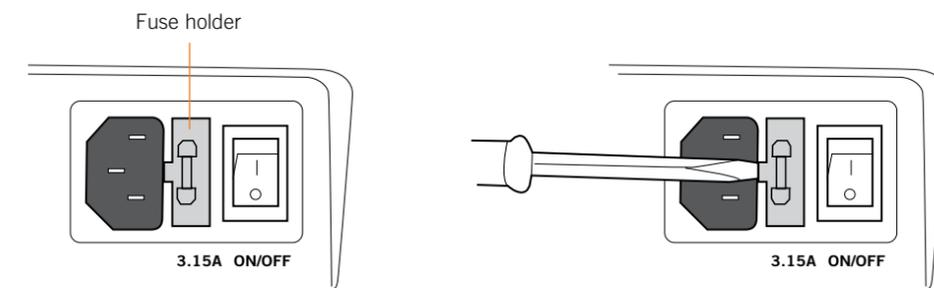
Fig.20

having trouble?

Your massage chair is a sophisticated combination of mechanics, electronics, and software working together to give you the ultimate massage. With such complexity comes opportunity for what we like to call massage chair “confusion.” So, should your massage chair exhibit any abnormal behavior, please do the following to fully reset the chair:

1. Press the tablet remote Power/Restore button to power off the tablet remote.
2. Turn the chair’s power switch (on the rear of the chair) to Off.
3. Unplug the chair from the power source.
4. Wait 30 seconds.
5. Plug the chair back into the power source, then turn the power switch back On.

If your chair has no power:

1. **Check the power cord** - Make sure the power cord connection to the chair is secure and that the power cord is not damaged.
2. **Check the wall outlet** - Plug a known working device into the outlet to ensure the outlet is working.
3. **Replace the fuse** - See page 21 for instructions.

If the issue remains, please do the following before contacting our customer experience team (this will help us help you more quickly):

1. Locate the serial number on label on the back of your chair and write it down.
2. Locate your proof of purchase, as we will need it to process any warranty claim.
3. Now you’re ready to contact us!

Having Trouble? Contact our Customer Experience team at **800-355-2762**,
Monday – Thursday 7 AM to 4:30 PM PST, Friday 7 AM to 3:30 PM
or **csadmin@humantouch.com**

Specifications*

Operating Voltage:	AC 110-120 V, 60 Hz
Power Consumption:	150 W
Charger Ratings:	USB @ 5 V, 3 A, Wireless @ 10 W
Chair Size:	61” L x 30” W x 44” H (upright) 71” L x 30” W x 33” H (reclined)
Product Weight:	180 lbs
Required Recline Clearance:	3”
Maximum Load Weight:	285 lbs

**Specifications are subject to change without prior notice.*

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