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Manufactured for:
DARCO® International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Phone 304.522.4883
www.darcointernational.com

DARCO® (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 8807 9228 0
www.darco.de

Made in China

WAL-IFU-21_Rev0321

DARCO®

Walker

Instructions For Use

| FX Pro™

| Air Traveler™

| HD Walker

EN

Indications

> Postoperative use, trauma care, stress fractures, soft tissue injury

Contraindications

> Open fractures, poor circulation

Cleaning Instructions/Maintenance

> When the shell or inner liner is slightly dirty, clean it with a damp cloth or a soft brush.
> Allow the shell or inner liner to air dry only.

Instructions for use - FX Pro™ and HD Walker

1. Open the hook and loop straps, remove the inner liner of the walker and take out the foam pads.
2. Wrap the liner around the foot and secure with the closures.
3. Spread open the stirrups and place the foot heel first into the walker.
4. With leg in walker and ankle positioned at 90°, affix the stirrups to the center of the liner on each side.
5. Place the foam pads in areas of excessive rubbing or to limit space (Optional).
6. Begin securing straps, starting at the toes and working towards the knee.

Instructions for use - Air Traveler™

1. Open the hook and loop straps, remove the inner liner of the walker and take out the foam pads.
2. Wrap the liner around the foot and secure with the closures.
3. Spread open the stirrups and place the foot heel first into the walker.
4. With leg in walker and ankle positioned at 90°, remove the white protective strip and affix the stirrups to the center of the liner on each side.
5. Place the foam pads in areas of excessive rubbing or to limit space (Optional).
6. Begin securing straps, starting at the toes and working towards the knee.
7. Inflate liner by making sure the release valve is tightened clockwise, do not over-tighten. Depress the gray bulb until foot and ankle are comfortably immobilized. Caution: DO NOT OVERINFLATE.
8. To release air pressure, rotate the release valve counterclockwise until the air begins to escape.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO® International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings or socks with the walker. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO® International, Inc.

to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

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