

CANDo[®] Athletic Tape



REF

24-4825 (1.5 in / 3.8 cm width roll, box of 32)

24-4826 (2 in / 5.1 cm width roll, box of 24)

Intended Use

Supports, compresses, and protects joints and muscles to help prevent sprains and strains during sports and physical activity.

Instruction for Use

1. Clean the parts of the body which need tape (do not use tape on open wounds).
2. Unwind tape and wrap around body part. Cut tape when body part is wrapped. Multi layer taping can be used to support the body for therapy.

Storage

Store in a dry, moisture proof place. Storage temperature of 23°F ~ 104°F (-5°C ~ 40°C) is highly recommended.

Shelf Life

2 years

Warning

Do not apply our tapes on wounds directly.

Disposal

Dispose the product after use complying with local regulations.



Fabrication Enterprises Inc
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



AJW Technology Consulting LTD,
4/4a Bloomsbury Square,
London WC1A 2RP, United Kingdom



MedEnvoy UK Limited
85, Great Portland Street, First Floor
London, W1W 7LT
United Kingdom



AJW Technology Consulting GmbH,
Breite Strasse 3
40213 Dusseldorf Germany



MedEnvoy
NL-4M-000000248
Prinses Margrietplantsoen 33,
Suite 123, 2595 AM,
The Hague (NL)



AJW Technology Consulting GmbH,
Düsseldorfer, Zweigniederlassung Zürich
Kreuzplatz 2, 8032 Zurich, Switzerland



MedEnvoy Switzerland
Gothardstrasse 28
6302 Zug
Switzerland

