

CANDo® Fixation Tape



REF 24-4845

Description

CanDo® Fixation Tape is made from spunlace non-woven fabric with a breathable mesh structure that allows the skin to breathe naturally, helping to wick away water and sweat while effectively reducing the risk of wound infection. The adhesive bandage provides reliable support to the affected body area while maintaining a flexible and comfortable stretch for improved mobility and comfort.

Intended Use

- Securely fix dressings, bandages, and medical devices to the skin
- Provide light support and stabilization for minor sprains, strains, or post-injury taping
- Hold gauze, wound dressings, or ostomy appliances in place
- Use in conjunction with CanDo® Rigid Strapping Tape for knee or shoulder taping techniques
- Adhere bandages or gauze against the skin

Instruction for Use

1. Clean and dry the skin completely before application
2. Tear off or cut desired length of tape. Use grid pattern on tape for precise application
3. Peel off adhesive backing and apply the tape smoothly to the target area without overstretching
4. Press firmly for better adhesion
5. Remove slowly and gently to avoid skin irritation

Storage

Store in a cool, dry place, away from direct sunlight and heat.

Shelf Life

2 years

⚠ Warning

Do not apply our tapes on wounds directly.

Disposal

Dispose the product after use complying with local regulations.



Fabrication Enterprises Inc
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



AJW Technology Consulting LTD,
4/4a Bloomsbury Square,
London WC1A 2RP, United Kingdom



MedEnvoy UK Limited
85, Great Portland Street, First Floor
London, W1W 7LT
United Kingdom

EC REP

AJW Technology Consulting GmbH,
Breite Strasse 3
40213 Dusseldorf Germany



MedEnvoy
NL-4M-000000248
Prinses Margrietplantsoen 33,
Suite 123, 2595 AM,
The Hague (NL)

CH REP

AJW Technology Consulting GmbH,
Düsseldorf, Zweigniederlassung Zürich
Kreuzplatz 2, 8032 Zurich, Switzerland



MedEnvoy Switzerland
Gothardstrasse 28
6302 Zug
Switzerland

