

CANDo®

Rigid Strapping Tape



REF 24-4846

Description

CanDo® Rigid Strapping Tape is a non-elastic adhesive medical tape made from breathable rayon fabric coated with a pressure-sensitive adhesive. It's designed to provide the mechanical support and stabilization of joints, ligaments, and muscles, and it can help restrict excessive or abnormal movement during physical activity or rehabilitation. For external use only.

Intended Use

- Provide support and stabilization for joints and soft tissue.
- Assist in injury prevention during physical activity.
- Help limit excessive joint movement; support during rehabilitation.
- Secure dressings or protective pads.
- Firm support for ankles, thumbs, wrists, hands, knees, and shoulders.
- Blister prevention — pre-tape feet during running and hiking.
- Plantar fasciitis support — strap arches to ease foot pain.

Instruction for Use

Preparation

1. Ensure skin is clean, dry, and free of oils or sweat.
2. Trim excessive body hair if needed.
3. Apply underwrap or padding for additional skin protection.

Application

1. Tear or cut the tape to the required length.
2. Apply smoothly over the target area.
3. Apply appropriate tension for the support required.
4. Avoid wrinkles or folds during application.
5. Press firmly for proper adhesion.

Removal

1. Remove slowly in the direction of hair growth.
2. Use medical adhesive remover if needed.

Storage

Store in a cool, dry place, away from direct sunlight and heat.

Shelf Life

2 years

⚠ Warning

Do not apply our tapes on wounds directly.

Disposal

Dispose the product after use complying with local regulations.



Fabrication Enterprises Inc
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



AJW Technology Consulting LTD,
4/4a Bloomsbury Square,
London WC1A 2RP, United Kingdom



MedEnvoy UK Limited
85, Great Portland Street, First Floor
London, W1W 7LT
United Kingdom

EC REP

AJW Technology Consulting GmbH,
Breite Strasse 3
40213 Dusseldorf Germany



MedEnvoy
NL-4M-000000248
Prinses Margrietplantsoen 33,
Suite 123, 2595 AM,
The Hague (NL)

CH REP

AJW Technology Consulting GmbH,
Düsseldorf, Zweigniederlassung Zürich
Kreuzplatz 2, 8032 Zurich, Switzerland



MedEnvoy Switzerland
Gothardstrasse 28
6302 Zug
Switzerland

