



spidertech

ENGINEERED FOR MOVEMENT

LOWER BACK SPIDER



Partner Application

ONE PIECE PRECUT
READY-TO-APPLY

Application Tips:



Ensure skin is dry and free from oils, lotions, and hair



Do not touch adhesive with your fingers



Do not stretch the tape when applying



Gently rub the tape after applying to activate the adhesive


Following these tips will ensure maximum benefits and extend the use of your product. Enjoy!

LOWER BACK SPIDER


Application instructions

Application Tips:


- Notice the tab numbers on the back side of the tape showing the order of application.
- Before you begin, bend and snap all the perforations.
- Ensure skin is clean, dry, & free from oils before application.

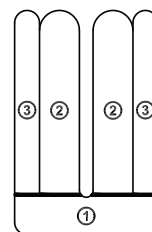

Partner Application

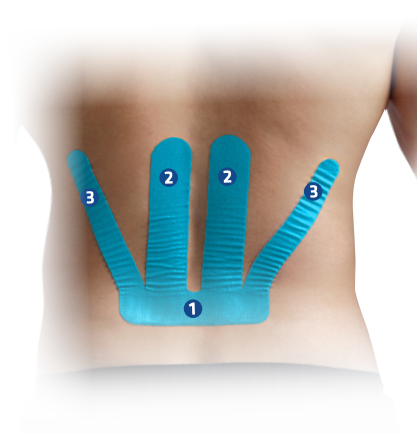

1

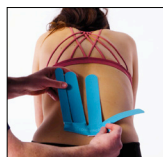

Remove the number 1 tab and apply the section at the bottom of the back, aligning the center of the spidertech logo with the spine. Rub tape to activate the adhesive.

2



Have the user bend forward and round their back as much as possible. Remove one of the number 2 tabs and apply the section to the skin, beside their spine.




Repeat this process with the other number 2 tab.

3




In the same bent forward position, align one of the number 3 tabs with their shoulder on the same side. Apply the section to the skin.

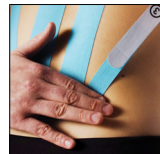


Repeat this process with the other number 3 tab.

4

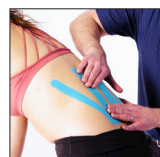

Take your time and gently rub the tape from the middle out to the ends to activate the adhesive.

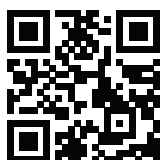

Watch our
how to apply
video here