

SMALL FAN
SPIDER



ENGINEERED FOR MOVEMENT

FAN SPIDER

LARGE FAN
SPIDER



MEDIUM FAN
SPIDER



ONE PIECE PRECUT
READY-TO-APPLY

Application Tips:



Ensure skin is dry and free from oils, lotions, and hair



Do not touch adhesive with your fingers



Do not stretch the tape when applying



Gently rub the tape after applying to activate the adhesive

Following these tips will ensure maximum benefits and extend the use of your product. Enjoy!

FAN SPIDER

Application instructions



Application Tips:


- Notice the tab numbers on the back side of the tape showing the order of application.
- Before you begin, bend and snap all the perforations.
- Ensure skin is clean, dry, & free from oils before application.

1




Remove the number 1 tab and apply the section at the top of the treatment site. Rub tape to activate the adhesive.

3

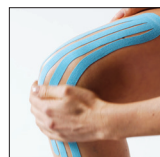

Take your time and gently rub the tape from the middle out to the ends to activate the adhesive.

2

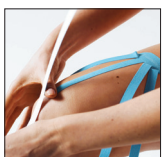

Working from the center out, remove the tab and apply the section over the treatment site.



Repeat this process with the other tabs. Ensure none of the tape overlaps.



Watch our
how to apply
video here