KINESIOLOGY TAPE



i-STRIPS

SNAP STICK










How To Apply:

- Apply i-STRIPS to clean or shaved skin.
- Avoid touching the tape with your fingers to ensure that the tape sticks to the targeted area.
- Fold along perforations, and then pull to **SNAP**.
- Peel back the backing paper on section 1.
- **STICK** to skin and gently rub the tape to activate the adhesive.
- Peel back the backing paper on section 2.
- Apply section ② and gently rub the tape to activate the adhesive.
- For best results, wait 30 minutes before exercise or contact with water.

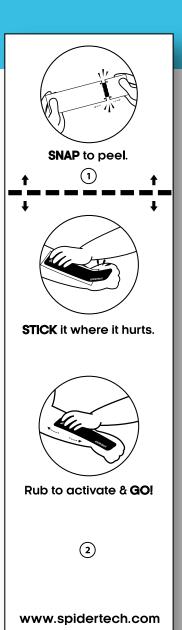

Common Applications:































KINESIOLOGY TAPE

SHOULDER

PAIN: Shoulder pain. Neck pain

BACK Strip 2

Pull arm horizontally across the body

1 Snap the tape backing and peel off section #1

Place section #1

Lay tape towards the bottom half of the shoulder blade. Rub to

FRONT Strip 1



Snap the tape backing and peel off section #1



Flex arm downward towards foot and bend neck towards opposite shoulder



Place section #1 just below shoulder joint, pointing up towards head



Peel backing towards head and lay and tape over the center of the shoulder, without stretching the tape



Rub to activate tape

NECK

Lean neck towards right shoulder

Place section at the center of neck with tape pointing down towards left shoulder

Rub to activate tape

muscles

PAIN: Neck sprains, tight neck

ELBOW

PAIN: Elbow pain. Wrist pain



Snap the tape backing and peel off section #1



Snap the tape backing and peel off section #1



Place tape on inner side of elbow joint, with palm up and wrist pointed down



Place tape on outside



Peel tape towards middle of palm, wrist



Peel tape towards middle of back of hand



Rub to activate tape



Rub to activate tape

KNEE PAIN: Knee pain



Position Hips at 45 degree angle and Knee at 90 degree angle



Snap the tape backing and peel off section #1



Snap the tape backing and peel off section #1



Place tape at the top of the kneecap, towards





Peel backing section #2 towards hips, without stretching the tape



Rub to activate tape



Place tape on skin and rub to activate

ANKLE

PAIN: Ankle sprains



Stretch the ankle



Strip 2



Snap the tape backing and peel off section #1





Place tape back of the leg



Peel tape backing down the leg under the heel



Place tape around the back of ankle



Rub to activate tape



PAIN: Low back pain



Snap the tape backing and peel off section #1



Bend forward from your hips and flex your low back forward



Place the tape at the bottom of your back (to the Left of spine), pointing up



Peel the tape backing towards your head. Apply without stretch





Repeat for right side of spine

How To Remove:

- Remove after 5 days of use
- Stretch the skin under the tape and gently roll off the tape
- Do not remove when the tape is wet

Peel the tape backing towards left shoulder without stretching

• Remove the tape in the direction of hair growth, away from the centre of the body and down the arms and legs

Please note that failing to follow tape application and removal guidelines can lead skin irritation and in some cases skin breakage

For application videos - visit spidertech.com/how-to-apply

spidertech 🚮 spidertechtape 🔽 @spidertech 💆

Produced By: SpiderTech Inc. 115 Ridgetop Road, Toronto, Ontario Canada M1P 4W9

Tel: 1 (866) 270 1753 Fax: (416) 490-4001

info@spidertech.com

