



Instruction Manual

Plastic Ball Chair (adult)

50cm




REF


30-1792
30-1792B
30-1792R
30-1792G

No	Parts	QTY
----	-------	-----

A		1
---	---	---

B		1
---	---	---

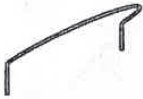C		1
---	---	---

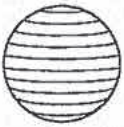D		1
---	---	---

E		2
---	---	---

F		1
---	---	---

G		1
---	---	---

H		1
---	---	---


No	Parts	QTY
----	-------	-----

I		4
---	---	---

J		2
---	---	---


K		1
---	---	---

STEP 1

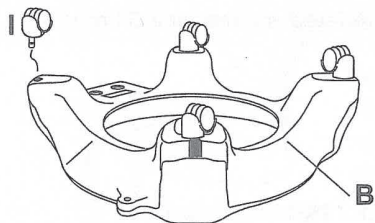


Begin by placing the base, labeled B, upside down on the floor, and installing the wheels, labeled I.

STEP 2

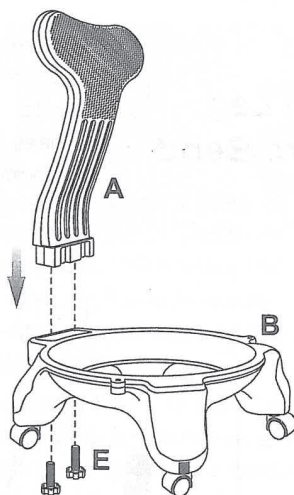

Turn the base over and insert the chair back labeled A into part B, locking it in place with the two knobs, part E


STEP 3

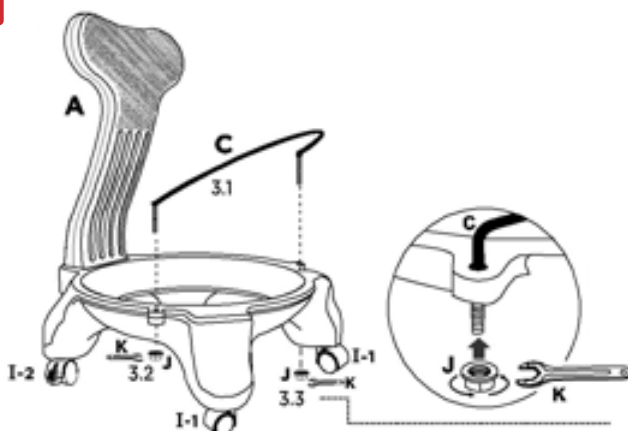

Install the plastic wire (C) into the base (B) and secure each side with the Nut (J). Use the wrench (K) to screw Nut (J) into the plastic wire (C) firmly.

STEP 4

To pump the ball, screw part F into the pump, labeled H, and inflate the ball until it is firm, then insert the plug, labeled G, which can be found taped to the ball.

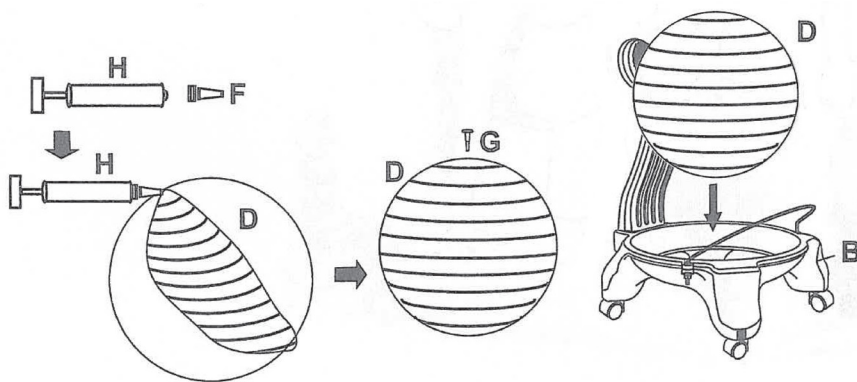


FINISHED

To pump the ball, screw the end piece into the provided pump, and inflate the ball until it is firm, then insert the plug, which can be found taped to the ball. The ball can now be placed onto the chair.

The material will expand to its full size 24 to 48 hours later, and you may finish inflating.

The ball may need to be topped off periodically over time.

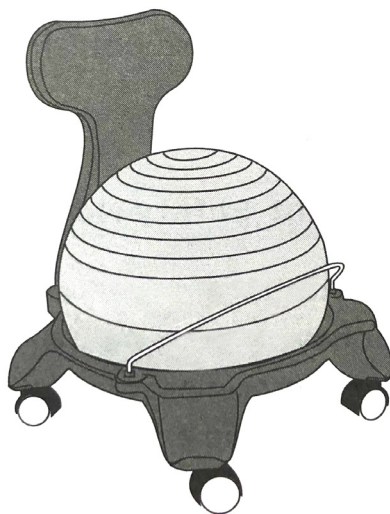

Fabrication Enterprises Inc.
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com

EC REP

AJW Technology Consulting GmbH,
Breite Strasse 3
40213 Düsseldorf Germany



MedEnvoy
NL-IM-000000248
Prinses Margrietplantsoen 33,
Suite 123, 2595 AM,
The Hague (NL)



CH REP

AJW Technology Consulting GmbH,
Düsseldorf, Zweigniederlassung Zürich
Kreuzplatz 2, 8032 Zürich, Switzerland



MedEnvoy Switzerland
Gothardstrasse 28
6302 Zug
Switzerland

UK
CA



AJW Technology Consulting Ltd
96 Theobalds Rd,
London, WC1X 8WB, United Kingdom

MedEnvoy UK Limited
167-169 Great Portland Street,
5th Floor, London, W1W 5PF
United Kingdom