ASPEN ACTIVE™ P-TLSO

THORACIC LUMBAR SACRAL ORTHOSIS

LUMBAL-SAKRAL-THORAKALE ORTHOSE

ORTESIS TORÁCICA LUMBOSACRA

ORTESI TORACICA SACRO-LOMBARE

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LUMBALE SACRALE THORACALE ORTHOSE

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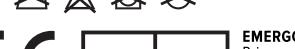
EMERG EUROPE
Prinsesgracht 20
2514 AP The Hague
The Netherlands

Pat. aspenmp.com/patents

Aspen Medical Products, LLC
6481 Oak Canyon, Irvine, CA 92618-5202 USA
aspenmp.com • +1-800-295-2776 • +1-949-681-0200
©2021 Aspen Medical Products, LLC. All Rights Reserved.
LB0003B revised 04/2021

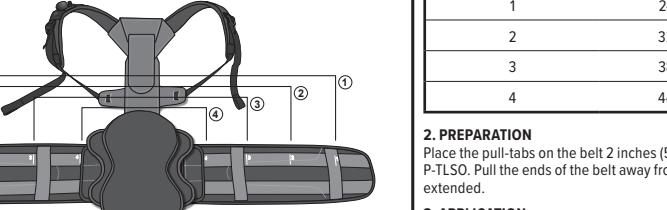
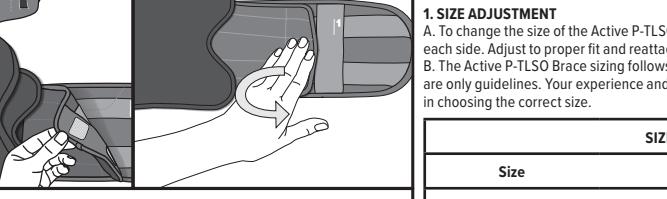
ENGLISH

DEUTSCH

ASPEN MEDICAL PRODUCTS GmbH

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aspenmp.de • +49 (0) 6155 89791-0 • Pat. aspenmp.de/patents

1. SIZE ADJUSTMENT
A. To change the size of the Active P-TLSO, peel up the inside end of the side panel on each side. Adjust to proper fit and reattach.
B. The Active P-TLSO Brace sizing follows closely to standard pants sizing. Size ranges are only guidelines. Your experience and personal assessment could make the difference in choosing the correct size.

SIZING GUIDE	
Size	Circumference
1	24 – 34 in 61 – 86 cm
2	32 – 40 in 81 – 102 cm
3	38 – 46 in 97 – 117 cm
4	44 – 50 in 112 – 127 cm

2. PREPARATION
Place the pull-tabs on the belt 2 inches (5cm) from the plastic in the back of the Active P-TLSO. Pull the ends of the belt away from each other ensuring that the belt is fully extended.

3. APPLICATION
A. Apply the fully extended brace, ensuring the back panel is centered and low on the back.
B. Apply the brace tightly around the abdomen, the right side overlapping the left.

NOTE: When applied properly, both the upper and lower edges of the brace should be snug. If there is any gap, reapply the brace and angle the ends to ensure a snug fit at both sides.

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C. Slip each thumb into the pull tabs. Extend the tabs out and away from the brace. When desired compression is achieved, place the hook side of the pull tabs down onto the loop material to secure.

4. ADJUSTMENT
A. Adjust the height of the shoulder straps. The yoke should sit at level C7 on patient.

B. Pull and tighten the shoulder straps until properly fit.

C. If shoulder strap length is too short or too long, adjust in the posterior section.

D. If the underarm strap placement is uncomfortable, adjust the posterior section as shown.

NOTE: After initial fitting, have patient sit, then stand, and re-adjust fitting as necessary.

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6. MODIFICATION OPTIONS
A. This product may be substantially modified by an individual with expertise. For more information, please visit www.aspenmp.com/education/product-modifications.

B. Recommended: Scan the QR code for a detailed instructional video.

INTENDED USE
To provide motion restriction and stability of the Thoracic-lumbar-sacral spine.

INDICATIONS
Postoperative support, Degenerative disc disease, Bulging or herniated disc, Fracture management, Spondylolisthesis, Spinal stenosis, Kyphosis, Spondylolisthesis (Osteoarthritis)/Degenerative Joint Disease.

CONTRAINDICATIONS
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

PRECAUTIONS
READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

CARE INSTRUCTIONS
Hand wash only at 30°C; Wash with mild detergent; Air dry; Do not use bleach or other chemicals.

MATERIALS
Nylon, Polyester, Polyurethane, ABS, Thermoplastic rubber, UHMWPE, HDPE, PVC, brass, Polycarbonate, Stainless steel

NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL
Product may be safely disposed in accordance with local laws.

REPORTING
Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

LIMITED WARRANTY
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or part(s). Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from an Aspen Medical Products, LLC or through its authorized channels of distribution;
(b) The product is altered;
(c) The product is not supplied by Aspen Medical Products, LLC or is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

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Aspen Medical Products, LLC ne sera pas obligé de remplacer ou de réparer les défaillances dans les cas suivants: