



ReOrder	Description	Size	U/M
554150	U-Turn Bolster	20"W x 19"H	Each

Purpose

The U-shaped base and curved interior slot design helps with turning the patient's body in order to prevent bedsore and muscle atrophy caused by prolonged bed rest. Keeps the lower extremities separated and in a stable position. Clinicians can use this device to check for pressure sores, change patient's absorbency pads or for temporary re-positioning.

- Easy to operate; reduces the burden on families and caregivers.
- Medical Grade Exterior is made of high-quality waterproof wipe clean fabric, which is comfortable, hypoallergenic, and easy to clean.
- The interior is a high-density foam, the foam is soft, supportive, and long-lasting provide perfect feel of comfort and support.
- U-Shaped Arc Design: Allows for easy rotation with decreased force exertion from the clinician.

Easy to Use

With the patient on the bed, place the turning device under the patient's leg/knee area, with a hand holding the handle, and the other hand holding the patient's waist or back area turn the patient's body towards you. To hold the patient in a turned position, slide the strap through the handle and secure the strap to the guardrail to stabilize the patient's body.