


Re-order #	Description	Size	Wt.	Unit
<input type="checkbox"/> 252000	Blue (Delrin Buckle)	2"x60"	.5 lbs	Each
<input type="checkbox"/> 252001	Blue (Delrin Buckle)	2"x60"	.5 lbs	24/pk
<input type="checkbox"/> 252002	Blue (Metal Buckle)	2"x60"	.5 lbs	Each
<input type="checkbox"/> 252003	Blue (Metal Buckle)	2"x60"	.5 lbs	24/pk
<input type="checkbox"/> 252004	Orange (Delrin Buckle)	2"x60"	.5 lbs	Each
<input type="checkbox"/> 252005	Orange (Delrin Buckle)	2"x60"	.5 lbs	24/pk
<input type="checkbox"/> 252006	Orange (Metal Buckle)	2"x60"	.5 lbs	Each
<input type="checkbox"/> 252007	Orange (Metal Buckle)	2"x60"	.5 lbs	24/pk
<input type="checkbox"/> 252250	Blue w/Handles (Delrin Buckle)	2"x48"	.5 lbs	24/pk
<input type="checkbox"/> 252252	Blue w/Handles (Delrin Buckle)	2"x60"	.5 lbs	24/pk

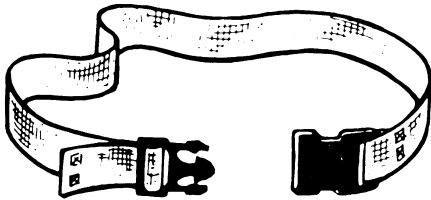

PURPOSE

The Gait Belt is intended for use when lifting, ambulating or transferring patients. It is not to be used as a safety restraint in a chair or bed.

USE INSTRUCTIONS

When an individual is at risk in ambulating without assistance gait belts should be used. The gait belt should be placed around the waist and when the buckle is closed the gait belt should fit snugly but not too tight to introduce resident discomfort.

The Delrin side release buckles are secured by inserting the male end of the buckle into the female side. To release the buckle, simultaneously press the tabs at the top and bottom of the buckle. Before ambulating the resident make certain that the buckle is positioned out of reach of the resident.

The resident should be ambulated by having the caregiver placing their hands through the belt palms up and grasping the belt. Before ambulating the resident, it should be confirmed that the number of caregiver's and their ability is sufficient for ambulation without introducing risk to either the caregiver or resident. The belt should be inspected before each use to assure that it isn't defective.

CARE INSTRUCTIONS

These devices may be wiped clean with a mild disinfectant. DO NOT LAUNDER