


Re-Order #	Size	Unit
911585	24" x 72"	Each
911590	30" x 72"	Each

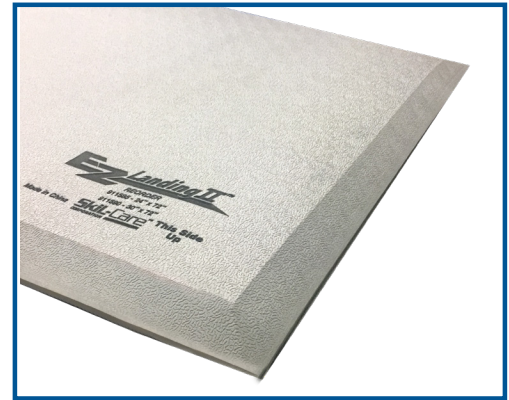

Beveled Low Profile Edges

Purpose

Skil-Care EZ Landing Fall Mats are designed to help reduce the impact from bed related falls. The one-piece electrometric foam surface is beveled to provide a low-profile edge to reduce tripping hazard. The anti-microbial agent protects the mat from mold, mildew and order causing bacteria. The low-profile surface allows resident to stand, walk or wheel over the mat without difficulty. The mat comes with a non-skid top and a slip resistant bottom.

Easy to Use

This mat is available in two widths and the choice of width should be made by a trained professional.

1. Place the mat in front of the bed spaced evenly between both legs and parallel with the bed (mattress).
2. Assure that the mat is placed with the beveled side up.
3. Although this product is durable, rolling the bed wheels over the mat should be avoided.
4. The mat could be stored flat or rolled loosely.

Easy Care

The EZ Landing II fall mat could be wiped clean with a spray and wipe cleanser. If disinfected, avoid using chlorine (bleach) chemicals.