

## Partitioned Suction Bottom Scoop Dish with Cover

### Instructions for Use

- Place on a smooth, flat surface and press down firmly to secure suction. Once secured, the plate stays in place while you fill it with food.



- When finished, attach lid tightly for storage or transport.



- To remove, gently pull up on the suction tab at the edge of the base to release the seal and lift the plate off the surface.



---

## Care Instructions

- Wash after each use; store in a dry, ventilated place.
- Dishwasher safe (top or bottom rack).
- Microwave safe up to 302°F / 150°C .
- Freezer safe.

## Safety & Materials

- Latex free
- BPA free
- PFAS free
- Made with food-safe materials



Fabrication Enterprises Inc  
250 Clearbrook Road, Suite 240  
Elmsford, NY 10523 (USA)  
tel: +1-914-345-9300 • 800-431-2830  
fax: +1-914-345-9800 • 800-634-5370  
FabEnt.com



AJW Technology Consulting GmbH  
Breite Strasse 3  
40213 Dusseldorf Germany



**MedEnvoy**  
**NL-IM-00000248**  
Prinses Margrietplantsoen 33, Suite 123  
2595 AM, The Hague (NL)